MONARK
LT2

OBS!

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NOTE!

Use of the product may involve considerable physical stress. It is therefore recommended that people who are not accustomed to cardiovascular exercise or who do not feel completely healthy, should consult a physician for advice.

When using a chest belt, keep in mind that electromagnetic waves can interfere with the pulse signal. Mobile phones and the like should therefore not be used near the bike when using a chest belt.

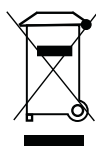
For service, maintenance and spare parts list, refer to our service manual. You can also contact your dealer.

Max user weight 180 kg.

Before using your new bike, please read the user information carefully and save it for future reference.


NOTE!

If you want to perform aerobic or anaerobic tests using this product, you should have relevant knowledge. We therefore refer to the relevant literature that describes the test procedure and analysis of measured results.


If the product or its packaging is marked with this symbol, it should not be handled as household waste. Instead, it shall be provided to a suitable collection point for the recycling of electrical and electronic equipment.

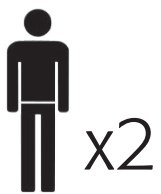
1)


2x

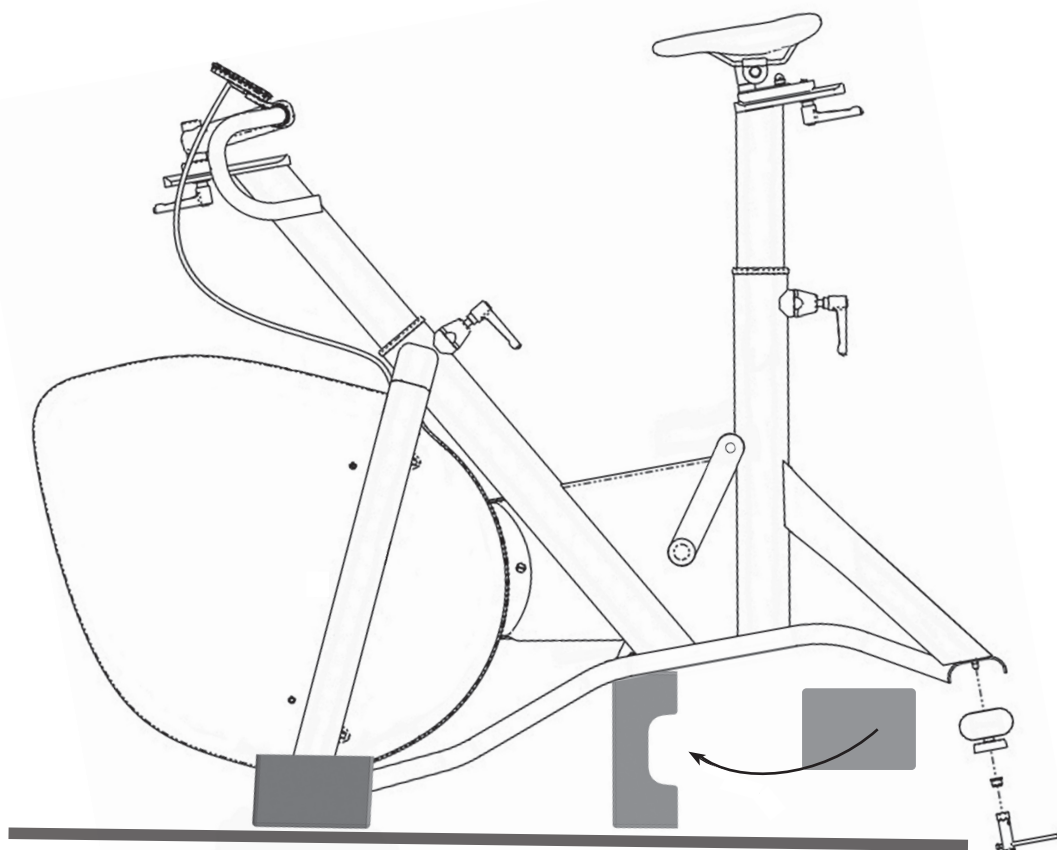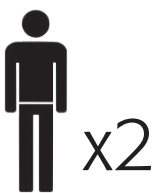2)

2x

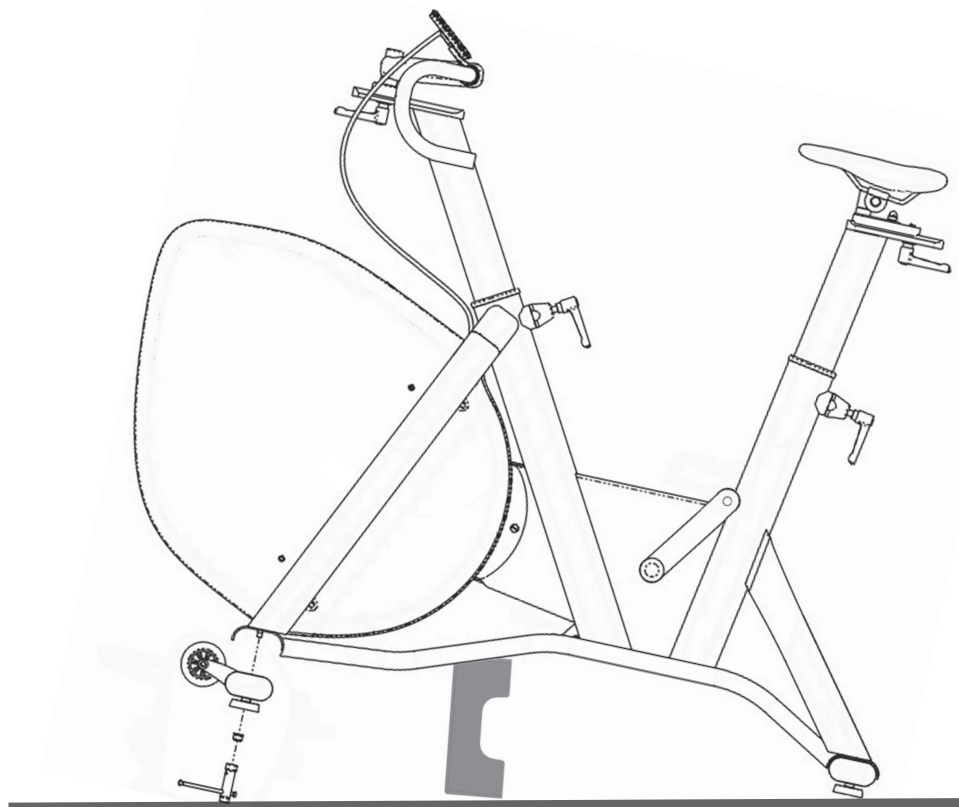
3)




NOTE!

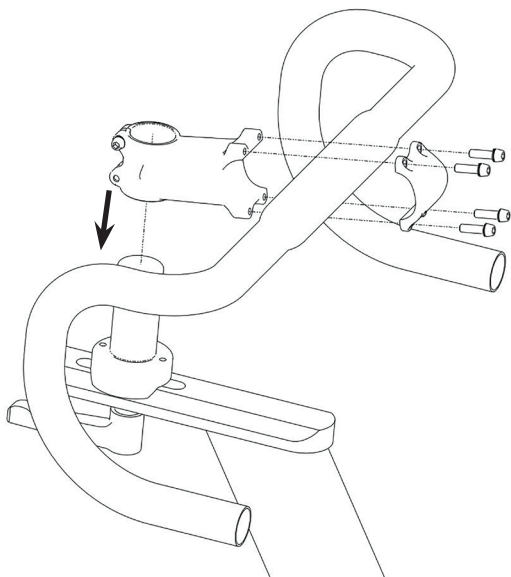The stem must be inserted at least 10 cm in the frame. This is marked with "MAX" on the stem.

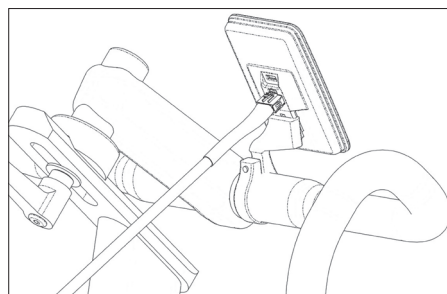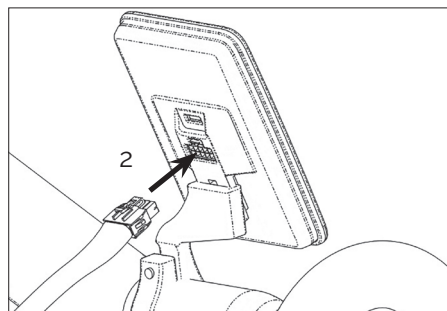
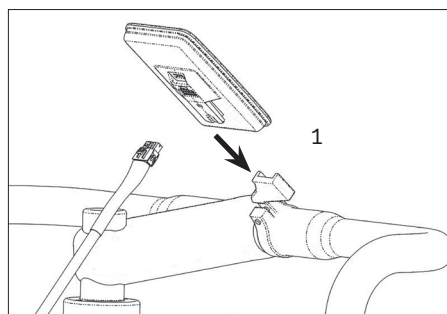
4)

5)

NOTE!

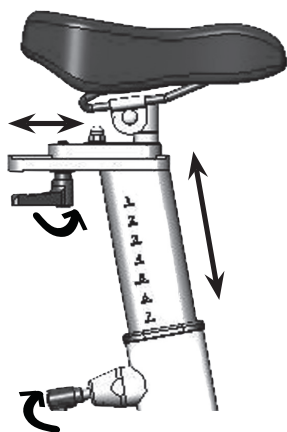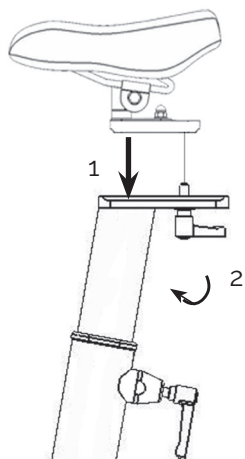The saddle post must be inserted at least 10 cm in the frame. This is marked with "MAX" on the seat post.

6)

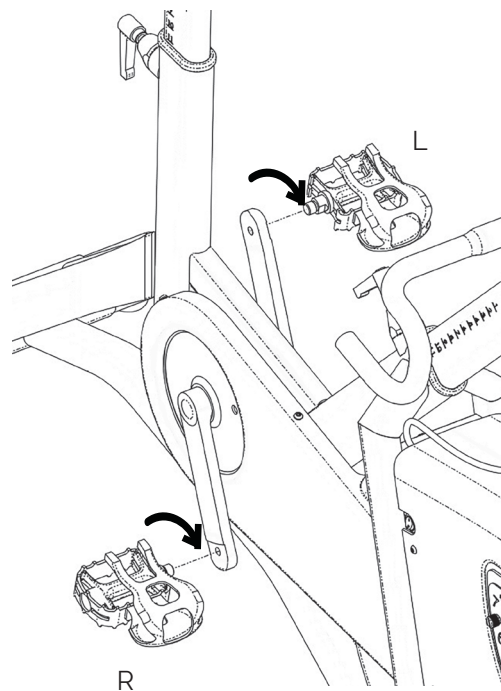
1x R

1x L

1x

NOTE!

When the bike is new, or if you replaced the pedals, they must be tightened after 5 hours of use.

DISPLAY

To start the display, press the on / off button on the right side of the bike, see fig: On / off.

Start the measurement by pressing "play"

Pause the measurement by pressing "pause"

Reset the device by pressing "stop"

RUN

STOP

PAUSE

SETTINGS

BACK

CONNECTED EXTERNAL DEVICE VIA ANT+

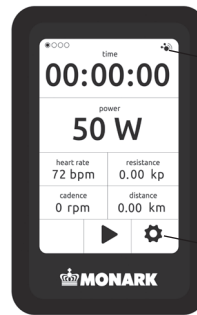
CONNECTED EXTERNAL DEVICE VIA BLE (BLUETOOTH)
(DOES NOT SUPPORT HEART RATE)


NOTE!

Do not expose the electronic display to direct sunlight or extremely high temperatures. Do not use solvents when cleaning, only a dry cloth.


ANT+ connected

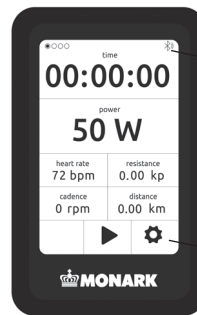Settings

BLE connected

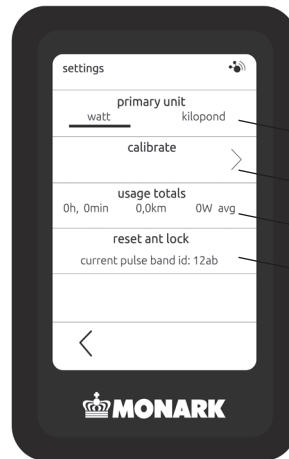Settings

Change units

Calibration mode

Total use, time, distance

Break lock to chestbelt

ANT+ / BLE

When the display is connected to an external device, the ANT+ / BLE symbol is displayed in the upper right corner.

Change units

It is possible to switch between kP and watt. You do this by going to "Settings" and selecting Kilopond or Watt as "primary unit".

Sleep mode

After ten minutes of inactivity, the display goes into sleep mode. To restart the display, press on the screen or start pedalling.

Display view

To change the display of data in the display, swipe your finger either to the right or to the left.

Fig: View 1

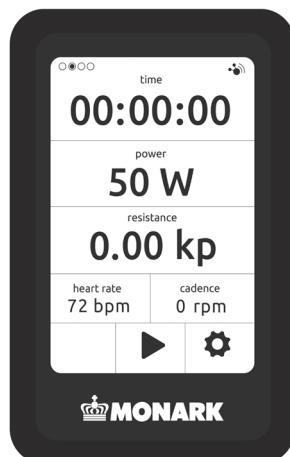
Fig: View 2

Fig: View 3

Fig: View 4

POWER SUPPLY

The bike is powered by a AC adaptor that is connected to a connection located on the right side of the bike, see fig: Connection.

Technical data AC adaptor

Input 100-240V AC 1.4-0.7A, 50-60Hz

Output 12V DC 5.0A

Polarity + in the middle

Fig: Polarity

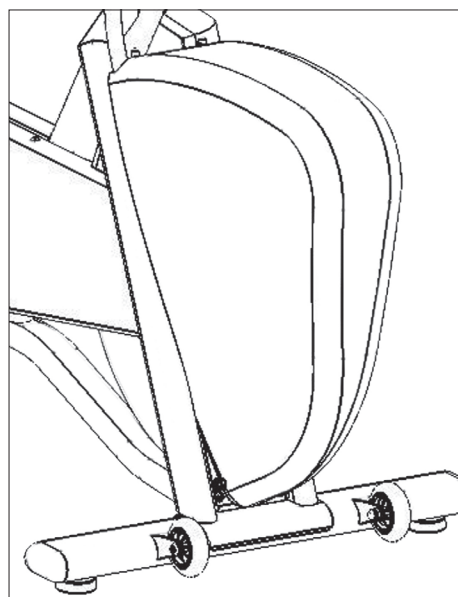
Fig:
Connection

WORKLOAD ADJUSTMENT

The workload is adjusted by using the control lever located on the handlebars. A is light workload, F is heavy workload.

Fine adjustment of the workload is done by the wire adjuster (2).

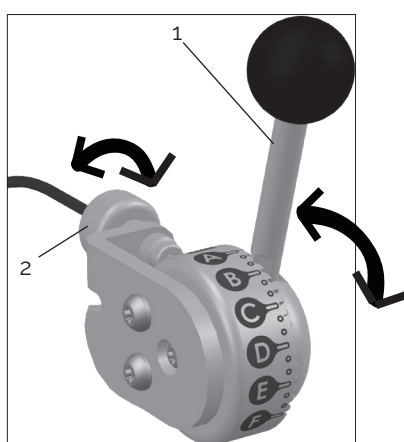
Fig: Workload adjustment

- 1) Workload lever
- 2) Wire adjuster

CALIBRATION

Calibration is necessary so the electronic and the mechanical parts of the cycle conform. We recommend that you do a mechanical calibration once a year and also after each service, changing of display and after moving. New calibration automatically replaces the old one.

- Before you can start the calibration, you must do the following:
- Set the workload control to the minimum load
- Remove the front cover
- Press down the pressure roller and lift off the wire from the black line wheel (below the potentiometer) so that the pressure arm goes up completely and loosens the brake band, see fig: Workload unit.


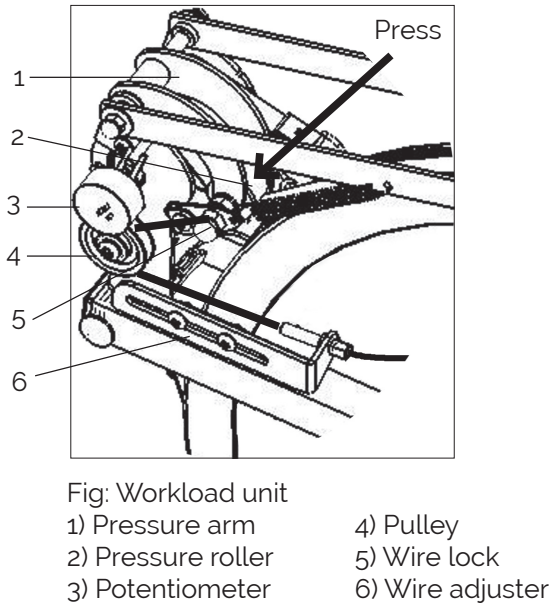
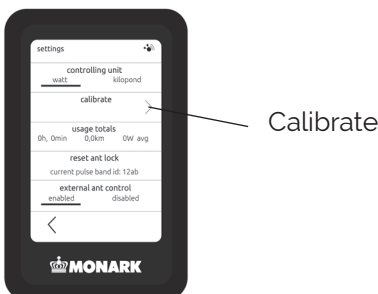
Fig: Workload unit

Start the calibration

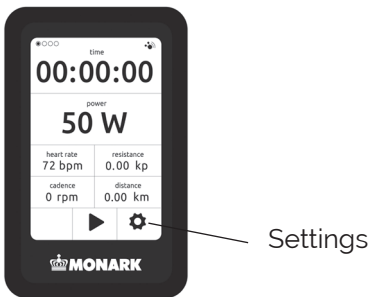After doing the above, you can start the calibration mode in the display. Tap Settings and then Calibrate. Then follow the instructions on the right.

NOTE!

To perform the calibration, you must have a 2 kg and a 4 kg calibration weight. These are available as accessories.

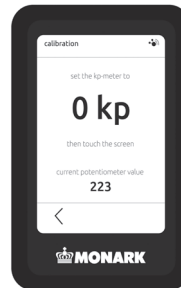
Fig: 0 kp

The first calibration point is at 0 kp. Then you use no weight at all. Tap the screen.

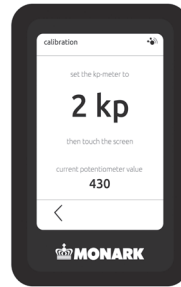
Fig: 2 kp

The next calibration point is at 2 kp. Then you hang in 2 kg calibration weight. Tap the screen.

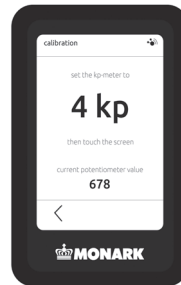
Fig: 4 kp

The next calibration point is at 4 kp. Then you remove the 2 kg weight and hang in the 4 kg calibration weight. Tap the screen.

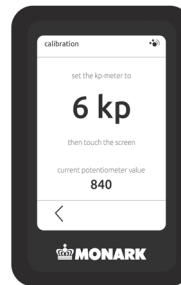
Fig: 6 kp

The last calibration point is at 6 kp. Then you use both 2 kg and 4 kg calibration weights. Tap the screen.

NOTE!

The potentiometer values shown in the pictures are only examples. The value varies depending on the value of the potentiometer at 0 kp.

HEART RATE FUNCTION

The user's heart rate can be measured with a chestbelt that detects the pulse of the heart. Chestbelt ANT+ is supplied as standard.

Heart rate monitoring requires that the chestbelt is correctly placed. Make sure that your skin is clean where the chestbelt should be placed. When it is correctly fitted the logo on the belt will be central and readable, outward and upright, by another person. The chestbelt should be secured at a comfortable tension around the mid section, just below the breast muscle, see Fig: Placement chest belt. Moisten the electrodes before use, see Fig: Moistening the electrodes.

Break lock to chestbelt

The display saves the last used chestbelt. This means that you do not need to connect to a chestbelt every time you ride the same bike. To break this go into "Settings" and press "reset ant lock" and the text changes to "no pulse..." Then the display can search for and lock to a new chestbelt.

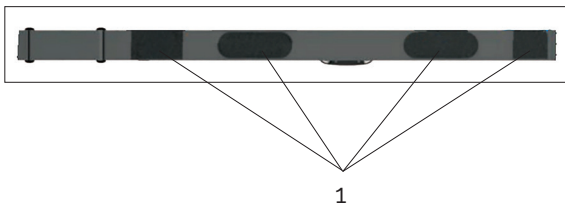
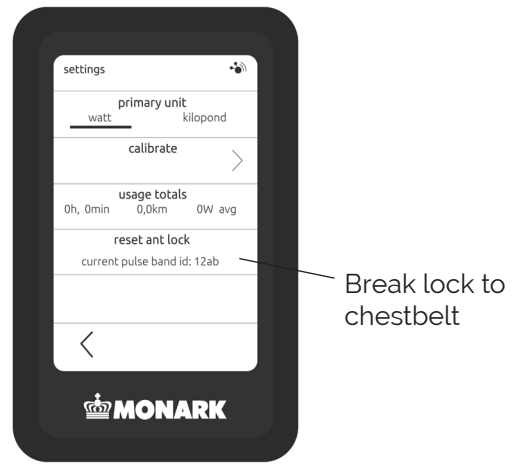
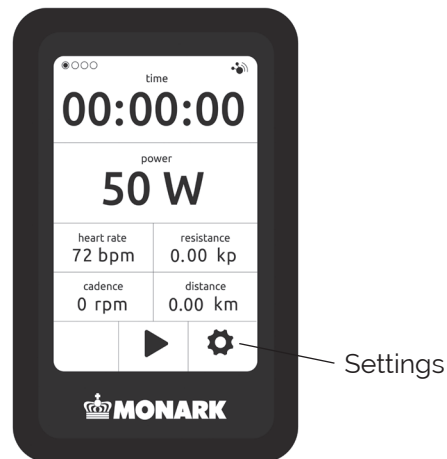
Fig: Moistening the electrodes (1)



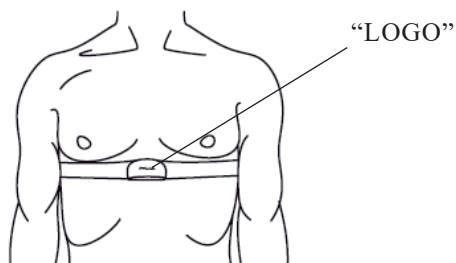
Fig: Placement chestbelt

NOTE!

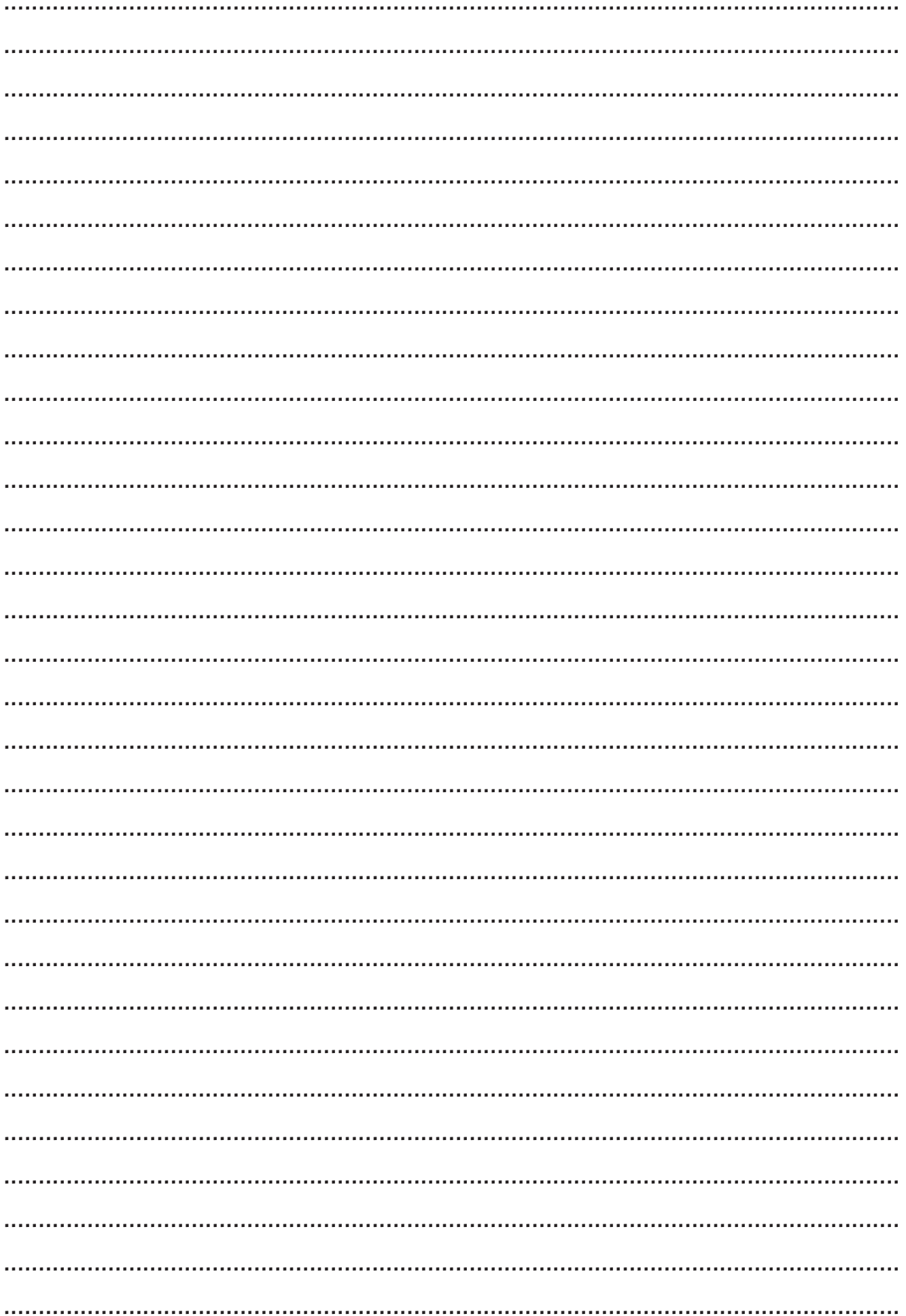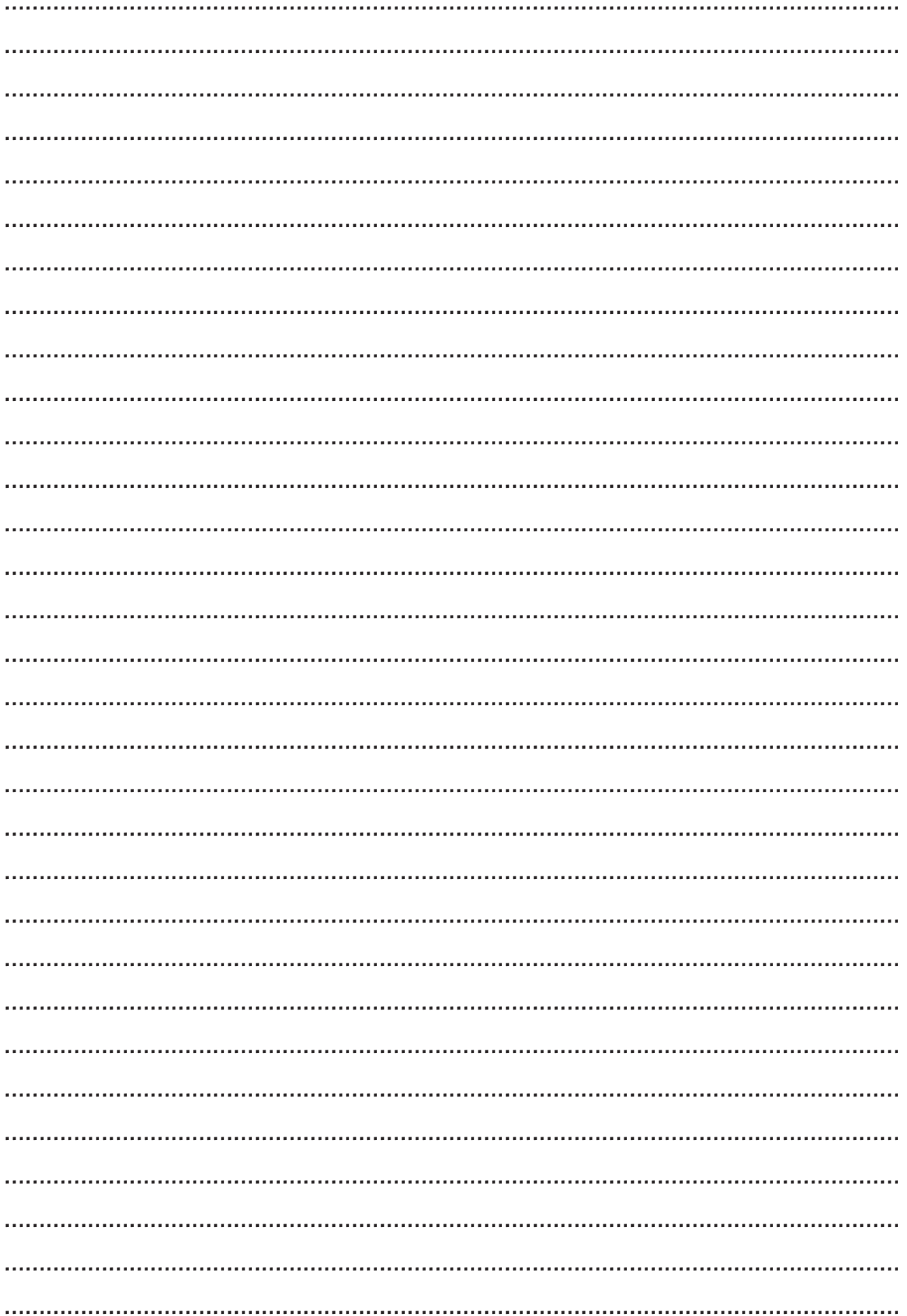Electromagnetic waves may disturb heart rate function. Cellular phones are not allowed to be used near the bike during test.

CE 0402

Version 2012

Art. Nr: 7950-379SE