**MONARK**  
**827 E**

**OBS!**

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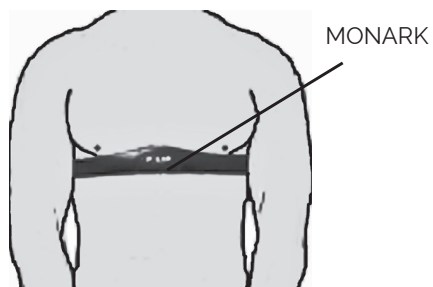
Fig:  
Placering pulsbalte

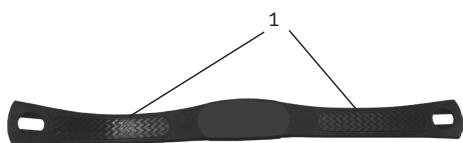
Fig:  
Fuktning av elektroder (1)

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A series of horizontal dotted lines intended for taking notes.

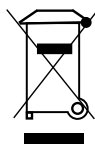
**NOTE!**

Use of the product may involve considerable physical stress. It is therefore recommended that people who are not accustomed to cardiovascular exercise or who do not feel completely healthy, should consult a physician for advice.

When using a chest belt, keep in mind that electromagnetic waves can interfere with the pulse signal. Mobile phones and the like should therefore not be used near the bike when using a chest belt.


For service, maintenance and spare parts list, refer to our service manual. You can also contact your dealer.

Max user weight 250 kg.


If the product or its packaging is marked with this symbol, it should not be handled as household waste. Instead, it shall be provided to a suitable collection point for the recycling of electrical and electronic equipment.

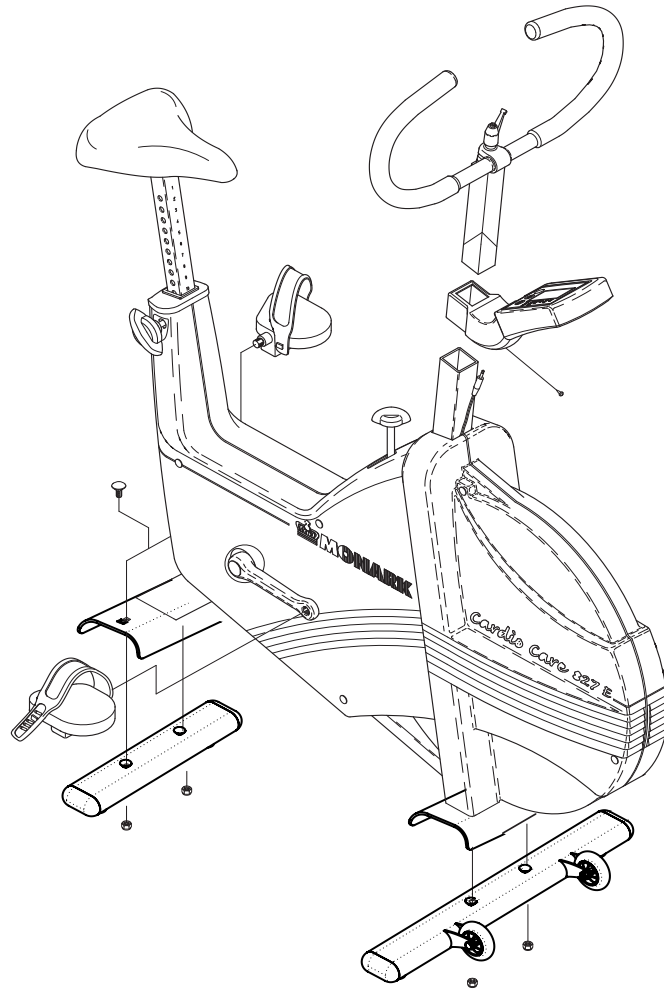
1)

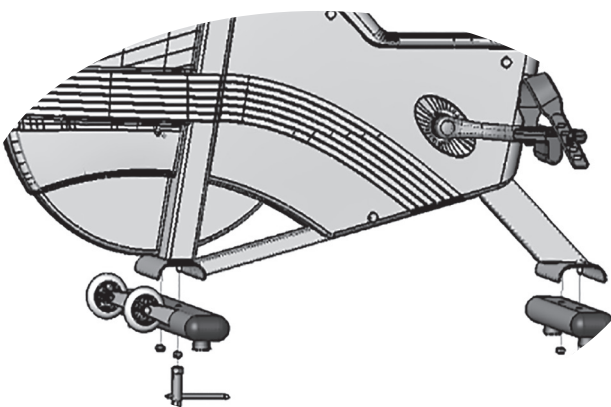2)

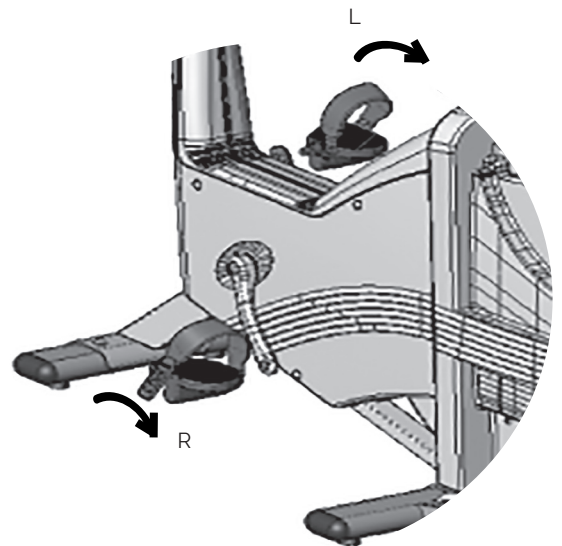
3)

**NOTE!**

When the bike is new, or if you replace the pedals, they must be tightened after 5 hours of use.

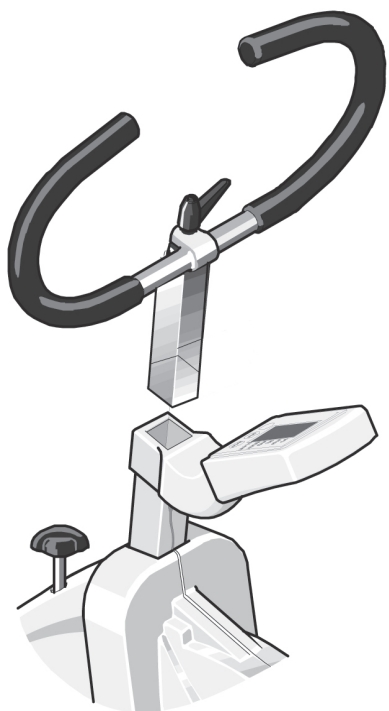4)



**NOTE!**

The stem must be inserted at least 8 cm in the frame. This is marked with "MAX" on the stem.


5)

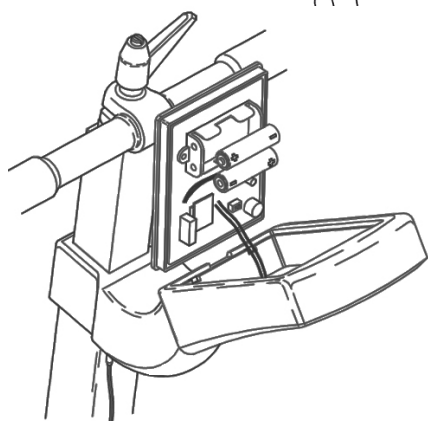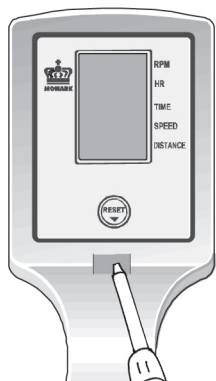6)

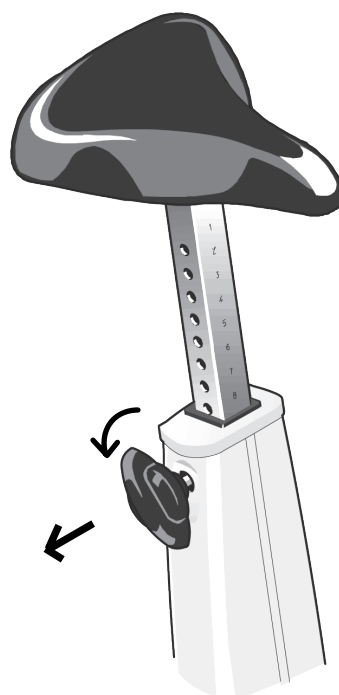7)

## DISPLAY

Press RESET button or start pedalling to start up the display. Then time, rpm, speed and distance counting. This can be reset by holding down the RESET button for more than two seconds.

At the HR window, a heart symbol lights up, which means that the display searches for a pulse signal. This is disconnected after 30 seconds if no pulse belt is found. You can start the search again by pressing the RESET button. When contacted, the heart symbol flashes in pace with the pulse rate.

### Replace batteries

The batteries are located under the display and it is easily opened by means of a chisel or the like, see Fig: *Battery replacement*.

### Change units

On the back of the display is a switch for displaying units in km / h and km or display in mph and miles. Then, remove the batteries (without using tools) and reinsert them.

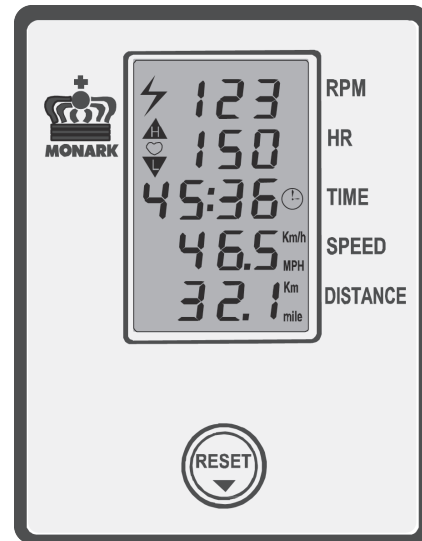
Fig:  
Electronic display

## WORKLOAD ADJUSTMENT

A change of the workload is done either by changing the pedal speed or by turning the tension knob to increase or decrease the brake belt tension on the flywheel.


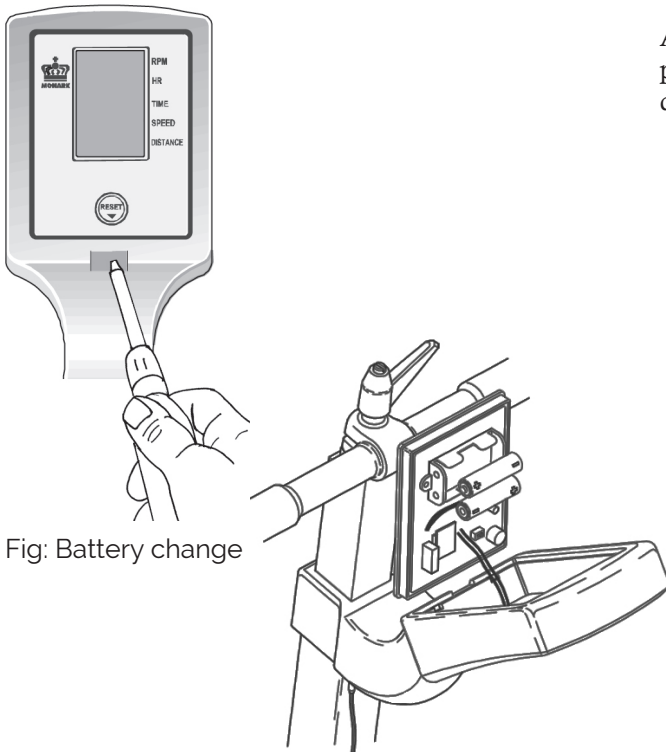
Fig: Battery change

### NOTE!

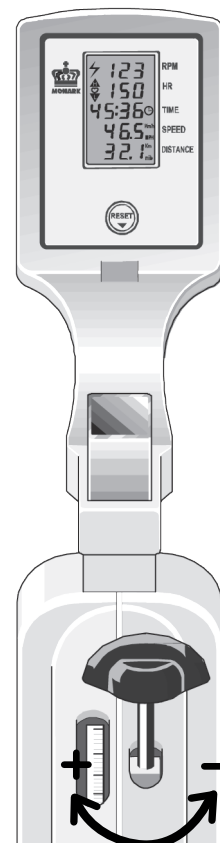Do not expose the electronic display to direct sunlight or extremely high temperatures. Do not use solvents when cleaning, only a dry cloth.

## HEART RATE

Chest belt is not included but available as an accessory.

In order for the heart rate measurement to work, it is necessary that the pulse belt is correctly positioned, see *Fig: Placement chest belt*. Moisten the electrodes before use, see *Fig: Moistening the electrodes*.

To make contact with the HR receiver on the bike, the distance should not be more than 100 cm. At the first usage it is important to first pair the chest belt with the display by standing close to it so the display can receive the pulse signal (max. 60 cm).

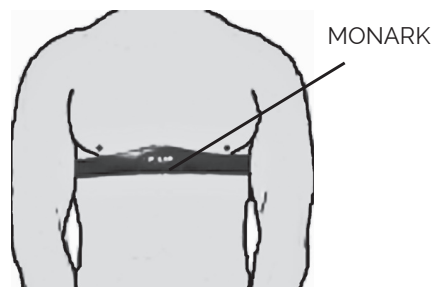
Fig:  
Placement chest belt

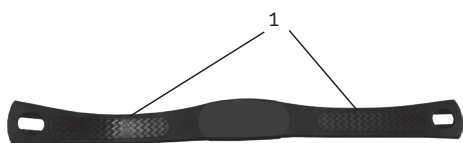
Fig:  
Moistening the electrodes (1)

# NOTES

A page of dotted lines for taking notes. The page contains 20 horizontal rows of evenly spaced dots, providing a guide for writing.

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