MONARK EXERCISE

FOR LIFE AND PERFORMANCE

SVENSKA
ENGLISH

MONARK
LC2

OBS!

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NOTE!

Use of the product may involve considerable physical stress. It is therefore recommended that people who are not accustomed to cardiovascular exercise or who do not feel completely healthy, should consult a physician for advice.

When using a chest belt, keep in mind that electromagnetic waves can interfere with the pulse signal. Mobile phones and the like should therefore not be used near the bike when using a chest belt.

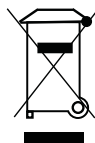
For service, maintenance and spare parts list, refer to our service manual. You can also contact your dealer.

Max user weight 180 kg.

Before using your new bike, please read the user information carefully and save it for future reference.

NOTE!

If you want to perform aerobic or anaerobic tests using this product, you should have relevant knowledge. We therefore refer to the relevant literature that describes the test procedure and analysis of measured results.

If the product or its packaging is marked with this symbol, it should not be handled as household waste. Instead, it shall be provided to a suitable collection point for the recycling of electrical and electronic equipment.

1)

**NOTE!**

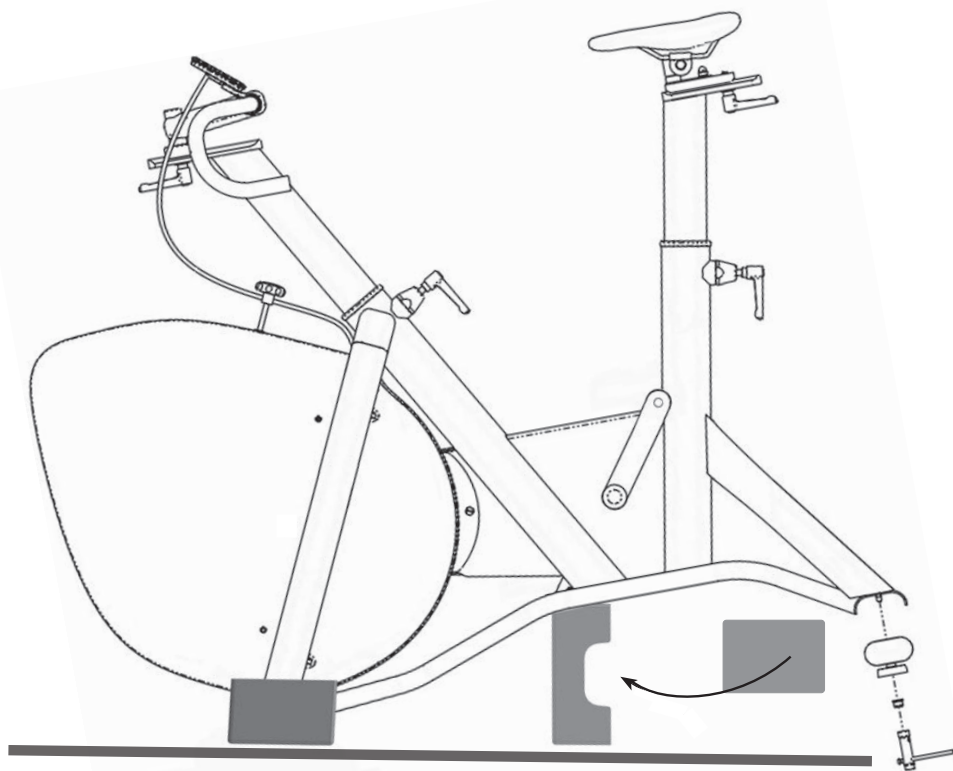
We recommend that you are two persons when the bike is being mounted.

1X

2X


2)

1X

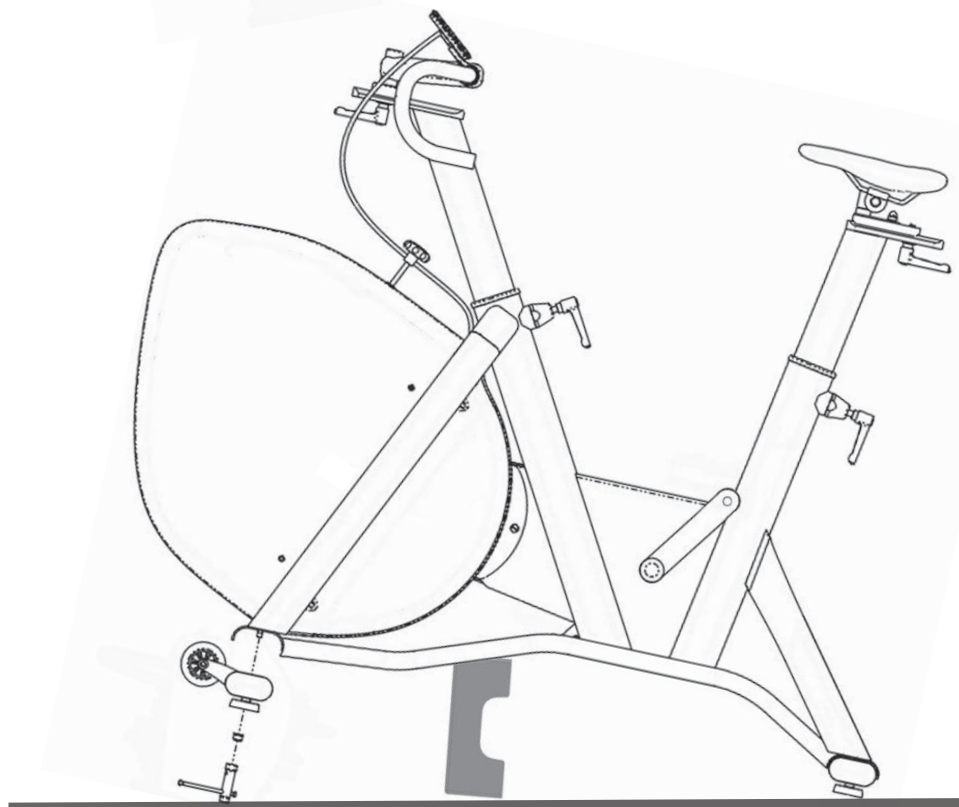2X


3)




NOTE!

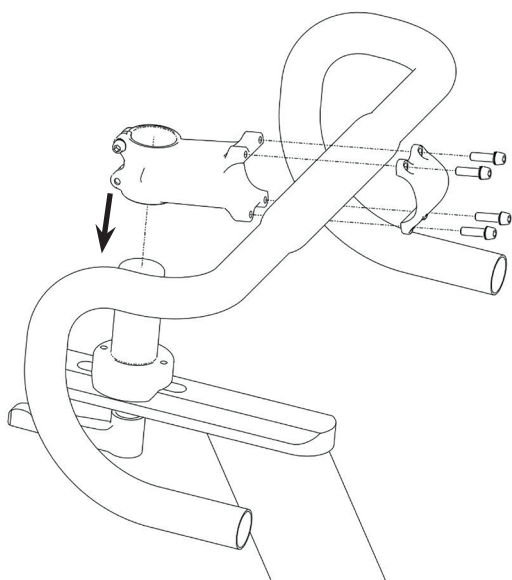The stem must be inserted at least 10 cm in the frame. This is marked with "MAX" on the stem.

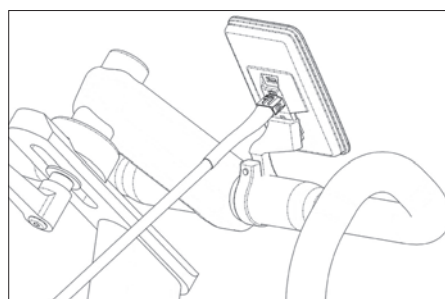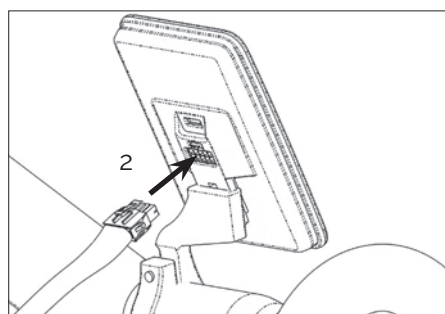
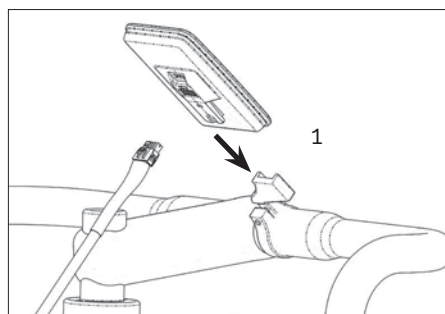
4)

5)

NOTE!

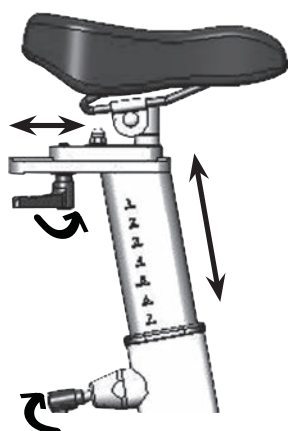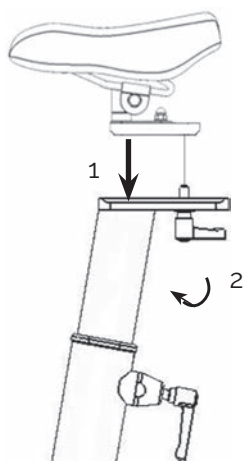The saddle post must be inserted at least 10 cm in the frame. This is marked with "MAX" on the seat post.

6)

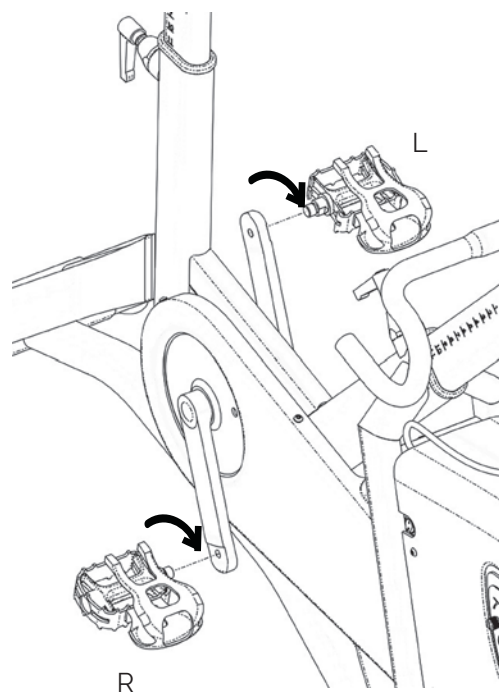
1x R

1x L

1x

NOTE!

When the bike is new, or if you replaced the pedals, they must be tightened after 5 hours of use.

DISPLAY

To start the display, press the on / off button on the right side of the bike, see fig: On / off.

Start the measurement by pressing "play"

Pause the measurement by pressing "pause"

Reset the device by pressing "stop"

RUN

STOP

PAUSE

SETTINGS

BACK

CONNECTED EXTERNAL DEVICE VIA ANT+

ANT+

When the display is connected to an external device, the ANT+ symbol is displayed in the upper right corner.

Change units

It is possible to switch between kP and watt. You do this by going to "Settings" and selecting Kilopond or Watt as "primary unit".

Sleep mode

After ten minutes of inactivity, the display goes into sleep mode. To restart the display, press on the screen or start pedalling.

Display view

To change the display of data in the display, swipe your finger either to the right or to the left.

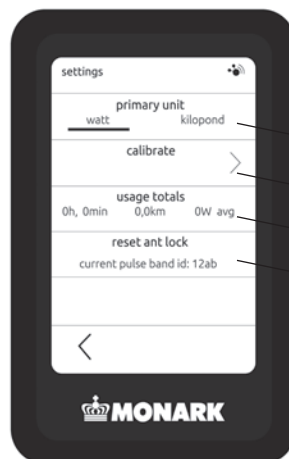
NOTE!

Do not expose the electronic display to direct sunlight or extremely high temperatures. Do not use solvents when cleaning, only a dry cloth.

ANT+ symbol

Settings

Change units

Calibration mode

Total use, time, distance

Break lock to chestbelt

Fig: View 1

Fig: View 2

Fig: View 3

Fig: View 4

POWER SUPPLY

The bike is powered by a power bank mounted under the front cover. The power bank is charged via a micro-USB connection located on the left side, see fig: Micro-USB connection. When you should charge the power bank you connect the supplied USB cable there. When fully charged, you can use the bike without having the power cord plugged in.

Power bank capacity

Lithium polymer, 3.6V, 2500mAh, 9Wh

Charging time 4 hours

Discharge time (without connected power cord): 1.5 hours depending on how you use the bike

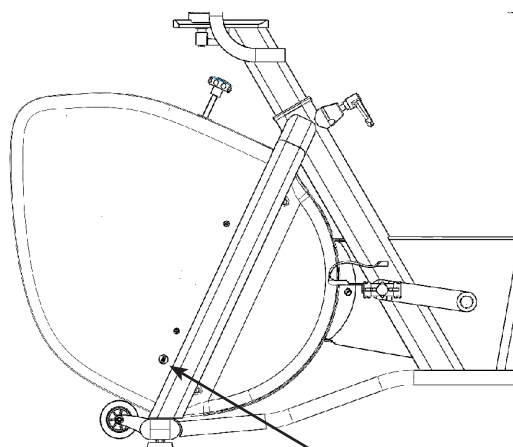
Fig: Micro-USB connection.


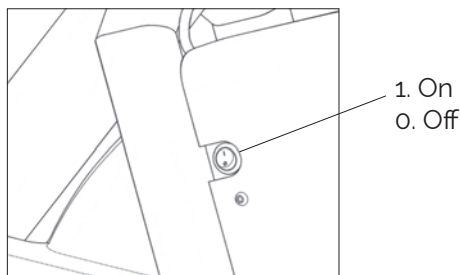
Fig:
On / off

WORKLOAD ADJUSTMENT

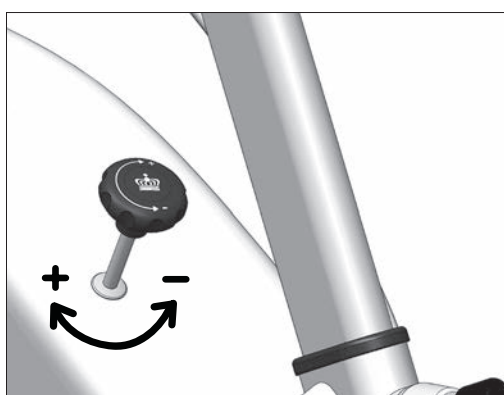A change of the workload is done either by changing the pedal speed or by turning the workload knob to increase or decrease the brake belt tension on the flywheel.

SCALE ADJUSTMENT

First, loosen the tension in the brake belt by screwing the workload knob counter-clockwise. See Fig: Release the tension in the brake belt.

Check that 0-index (3) on the scale (2) is in line with the pendulum pointer when the bike is at a stand still and the brake belt is loose. If not, the scale must be adjusted. Loosen the pole screw and adjust the scale. Tighten the pole screw (1) after adjustment. See Fig: Scale adjustment.

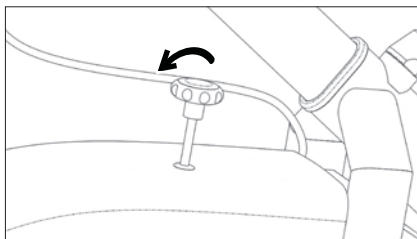
Fig:
Release the tension in the brake belt

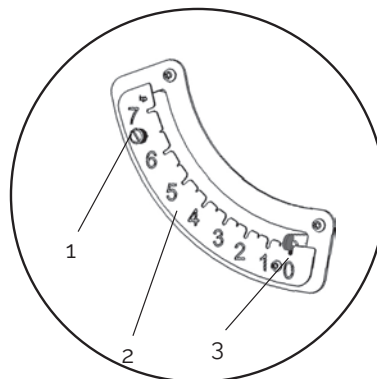
Fig: Scale adjustment
1) Screw
2) Kp-scale
3) 0-index

CALIBRATION PENDULUM

All test- and exercise cycles are calibrated in the factory, but a calibration of the pendulum can still be done to verify this.

Check the calibration of the pendulum once a year or when needed.

NOTE!

When calibrating the pendulum you need a 4 kg calibration weight.

The flywheel must be completely stopped before the weight is hung on!

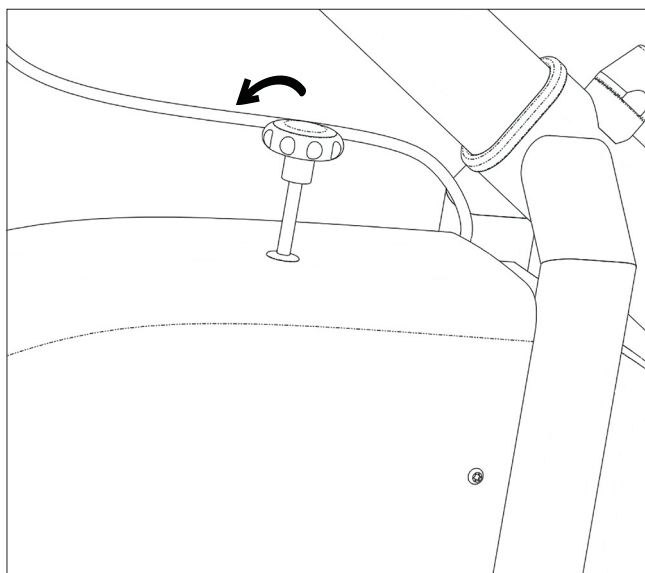
Fig 1:
Remove the load knob completely
(screw counter-clockwise).

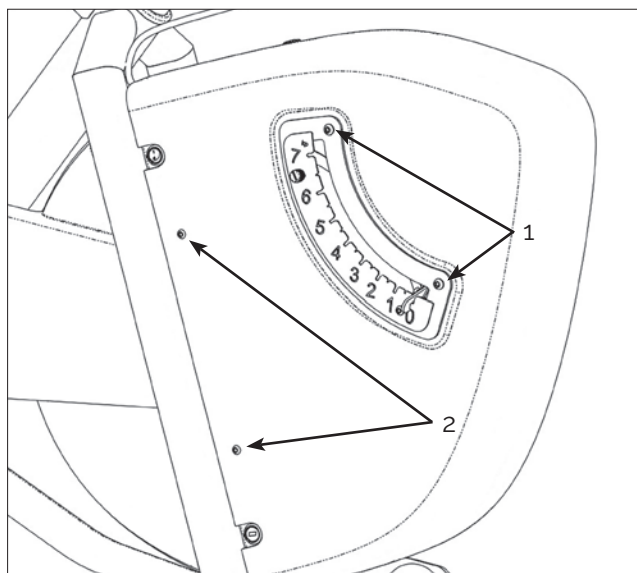
Fig 2:
1) Remove the scale
2) Remove the screws that holds the cover on the right side

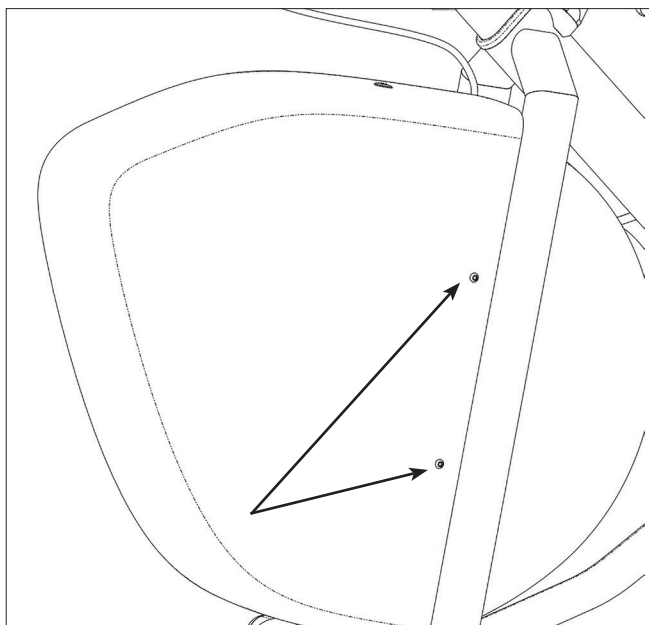
Fig 3:
Remove the screws that holds the cover on the left side

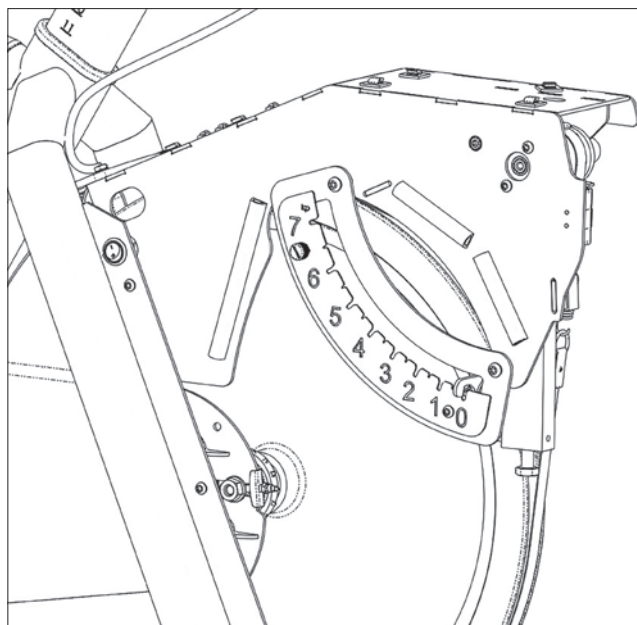
Fig 4:
After removing the front cover, you must screw back the scale again. Check again that the pendulum is at 0. If not you must reset the scale, see "Scale adjustment".

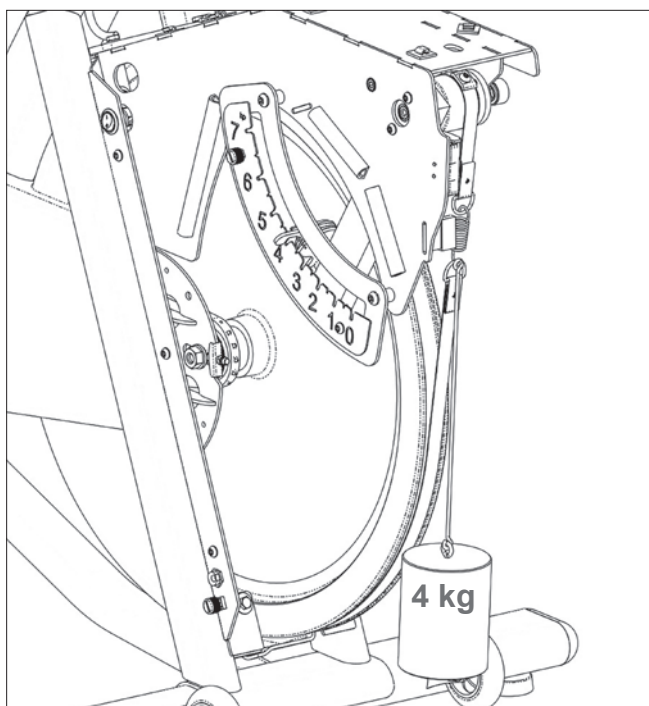
Fig 5:
Hang 4 kg calibration weight in the spring. Then the pendulum pointer should stay at 4 kp on the scale. If is not, the adjustment weight in the pendulum must be adjusted, see *fig*: Screw for adjustment weight in the pendulum.
If the pendulum goes too high, the adjustment weight must be moved outwards, if the pendulum goes too low, the weight must be Moved inwards. Repeat until the pendulum stops at 4 kp.

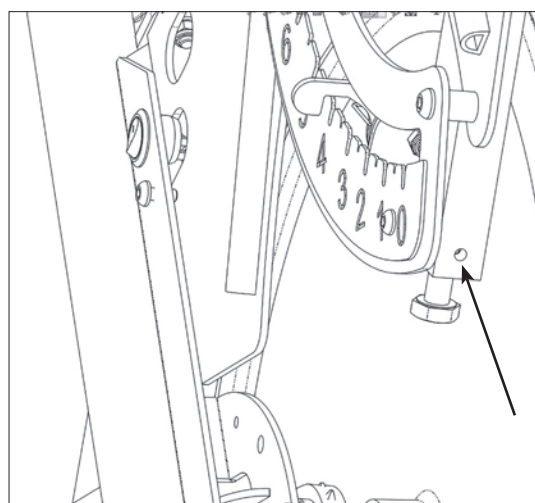
Fig:
Screw for adjustment weight in the pendulum.

When finished, remove the weight and replace the brake belt in the spring. Make sure that the brake band is on the surface of the brake wheel and has not ended up next to it. Then you unscrew the scale, reassemble the front cover, scale and finally screw back the load knob.

After calibrating the pendulum, you must do an electronic calibration, see section.

CALIBRATION ELECTRONICS

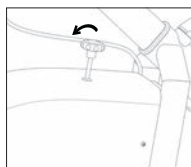
Calibration can only be done from the display!

Calibration should be done after service, replacement of electronics part, if you have moved the bike or if you have adjusted the scale.

Start by adjusting the scale, see section, and release the brake band tension.

1. Press "Settings" on the display. Then you select "calibrate".
2. Then "0 kp" is shown in the display. Hold the pendulum to 0 and then press the screen.
3. Then "2 kp" is shown in the display. Hold the pendulum to 2 and then press the screen.
4. Then "4 kp" is shown in the display. Hold the pendulum to 4 and then press the screen.
5. Then "6 kp" is shown in the display. Hold the pendulum to 6 and then press the screen.

The calibration is complete!


NOTE!

The potentiometer values shown in the pictures are only examples. The value varies depending on the value of the potentiometer at 0 kp.

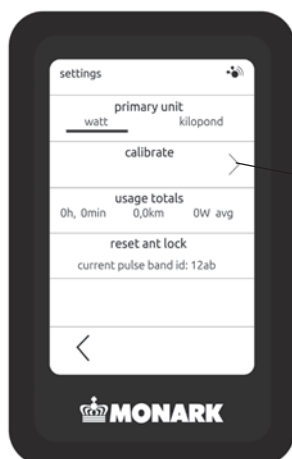Settings

Calibrate

NOTE!

The pendulum must be kept still at the different positions. This is done by pressing down the pointer into the groove on the scale at each kp value.

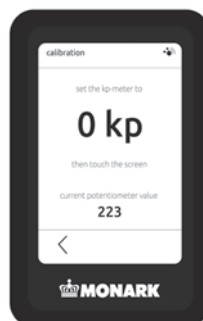
Fig: 0 kp

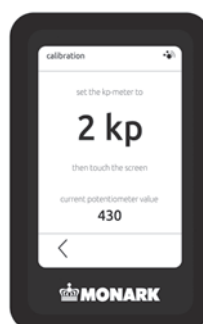
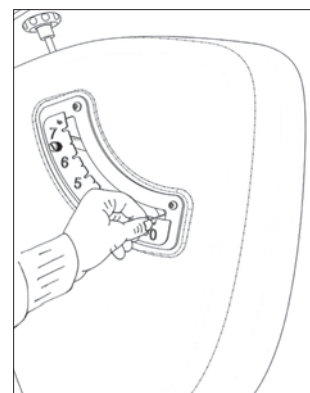
Fig: 2 kp

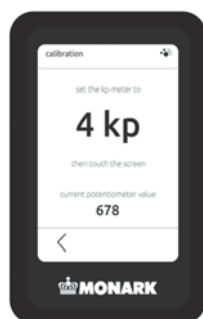
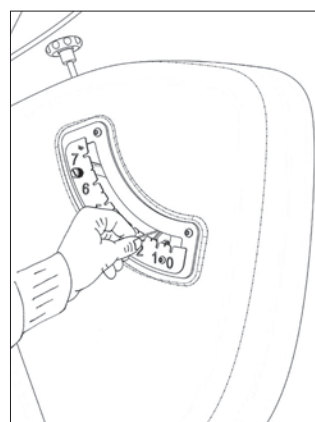
Fig: 4 kp


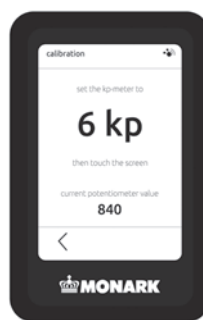
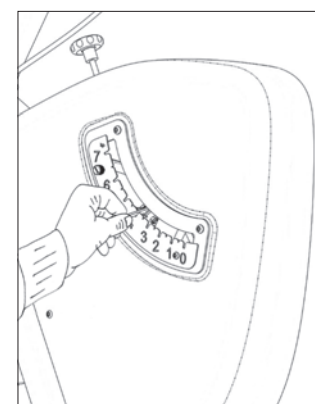
Fig: 6 kp

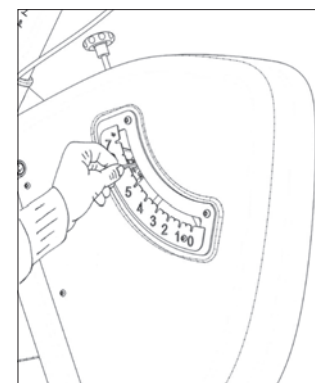Then move the pendulum down to 0 again and the calibration is complete.

HEART RATE FUNCTION

The user's heart rate can be measured with a chestbelt that detects the pulse of the heart. Chestbelt ANT+ is supplied as standard.

Heart rate monitoring requires that the chestbelt is correctly placed. Make sure that your skin is clean where the chestbelt should be placed. When it is correctly fitted the logo on the belt will be central and readable, outward and upright, by another person. The chestbelt should be secured at a comfortable tension around the mid section, just below the breast muscle, see Fig: Placement chest belt. Moisten the electrodes before use, see Fig: Moistening the electrodes.

Break lock to chestbelt

The display saves the last used chestbelt. This means that you do not need to connect to a chestbelt every time you ride the same bike. To break this go into "Settings" and press "reset ant lock" and the text changes to "no pulse...." Then the display can search for and lock to a new chestbelt.

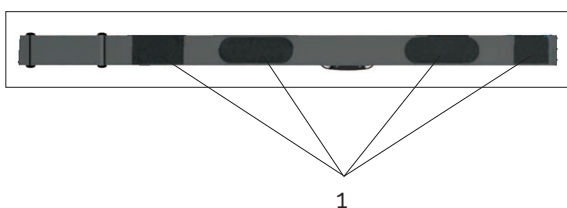
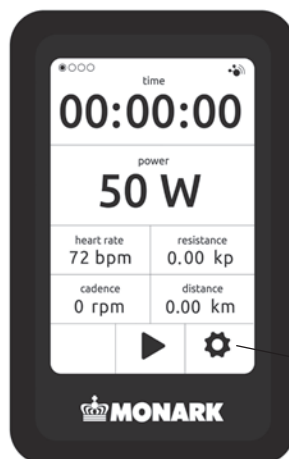
Fig: Moistening the electrodes (1)


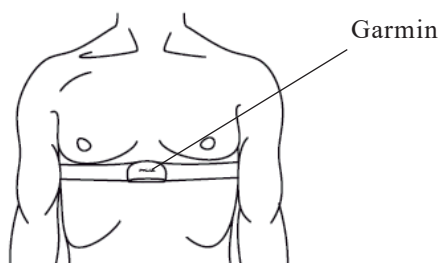
Fig: Placement chestbelt

NOTE!

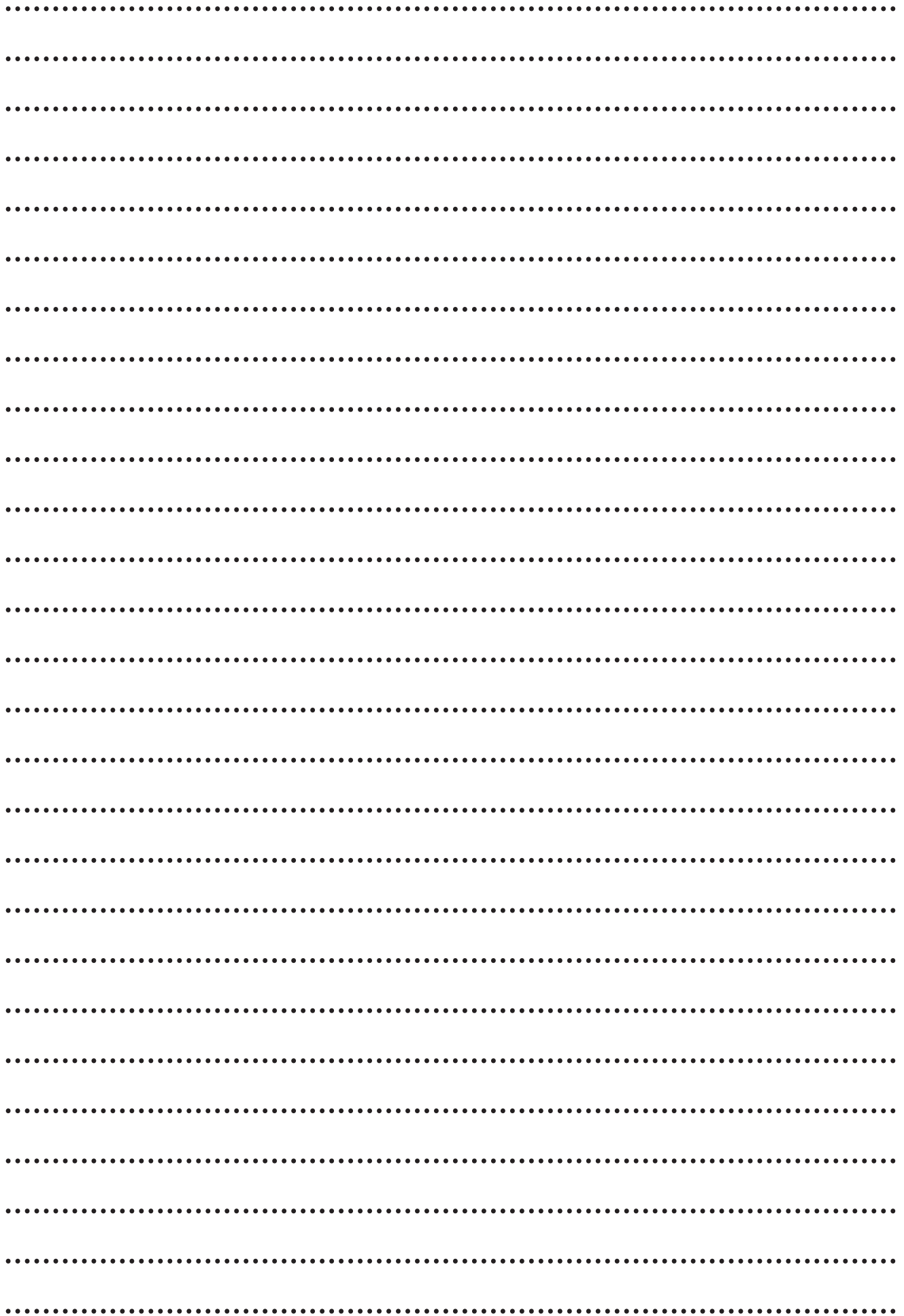Electromagnetic waves may disturb heart rate function. Cellular phones are not allowed to be used near the bike during test.

MONARK EXERCISE

FOR LIFE AND PERFORMANCE

Version 2026
Art. Nr: 7950-500SE