MONARK EXERCISE

FOR LIFE AND PERFORMANCE

SVENSKA
ENGLISH

MONARK LC7TT novo

OBS!

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NOTE!

Use of the product may involve considerable physical stress. It is therefore recommended that people who are not accustomed to cardiovascular exercise or who do not feel completely healthy, should consult a physician for advice.

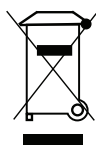
When using a chest belt, keep in mind that electromagnetic waves can interfere with the pulse signal. Mobile phones and the like should therefore not be used near the bike when using a chest belt.

For service, maintenance and spare parts list, refer to our service manual. You can also contact your dealer.

Max user weight 180 kg.




NOTE!

If you want to perform aerobic or anaerobic tests using this product, you should have relevant knowledge. We therefore refer to the relevant literature that describes the test procedure and analysis of measured results.


If the product or its packaging is marked with this symbol, it should not be handled as household waste. Instead, it shall be provided to a suitable collection point for the recycling of electrical and electronic equipment.

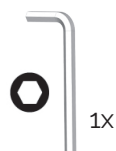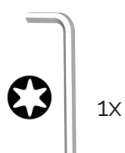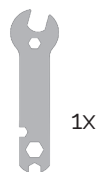1)


NOTE!

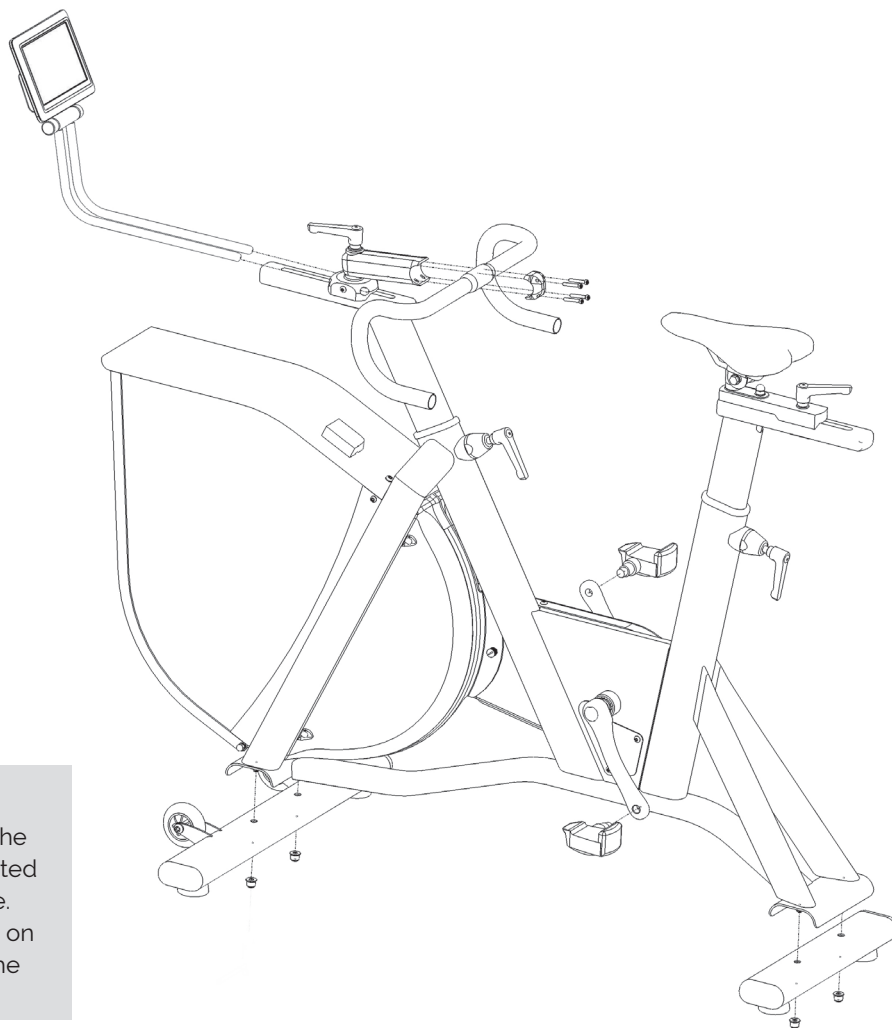
The handlebar stem and the saddle post must be inserted at least 10 cm in the frame. This is marked with "MAX" on the handlebar stem and the saddle post.





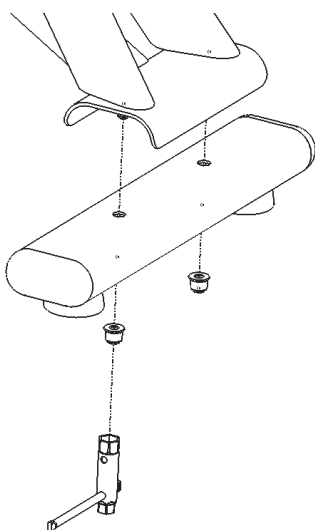
2)

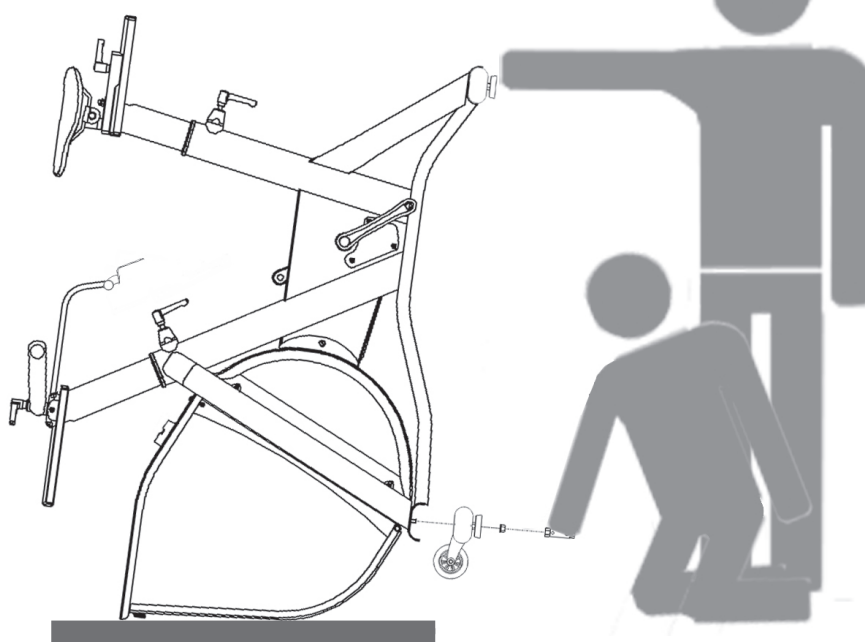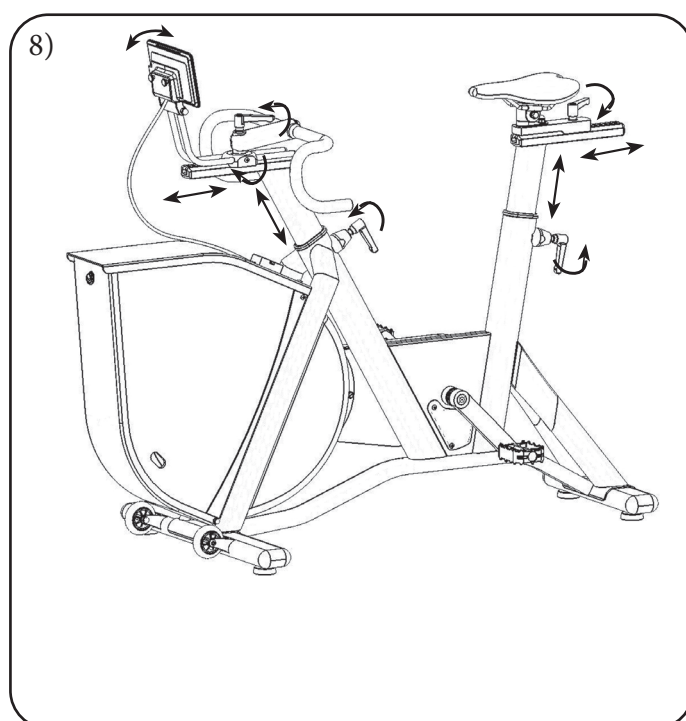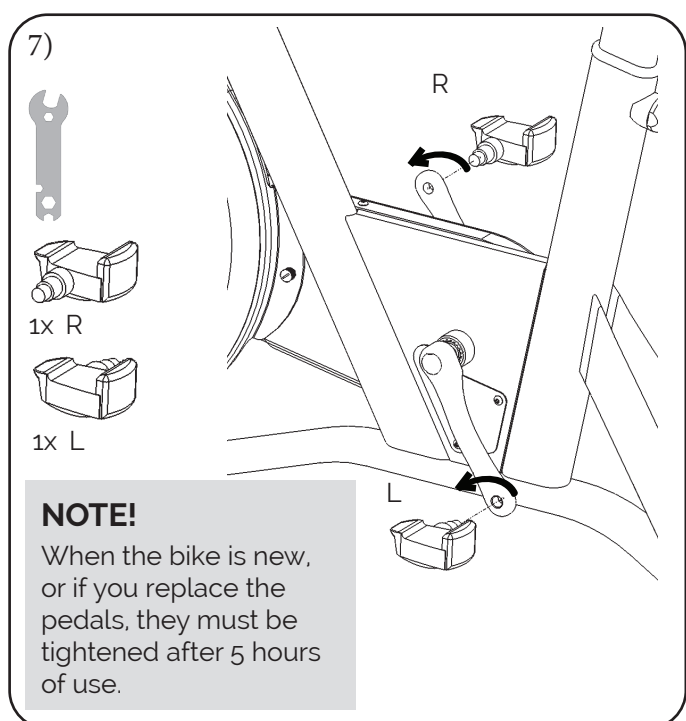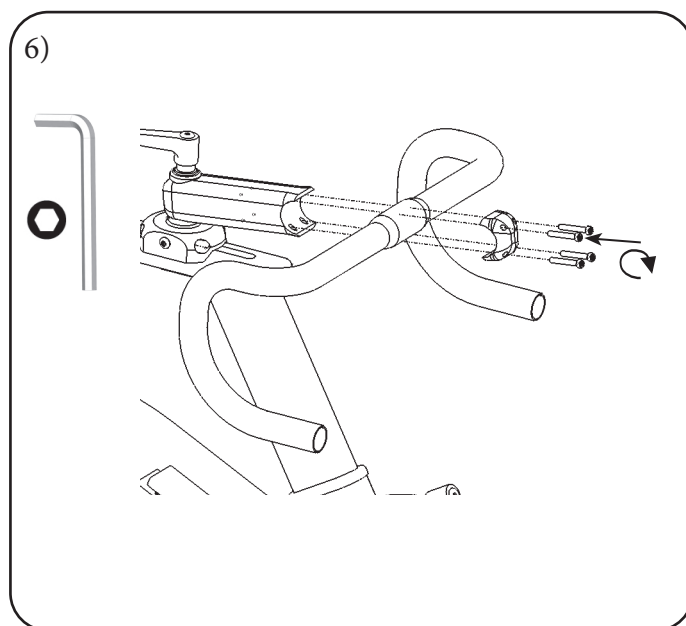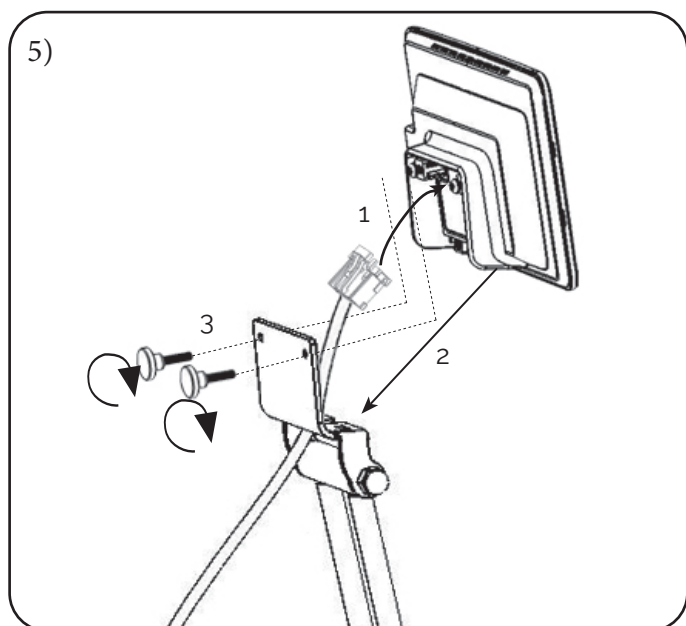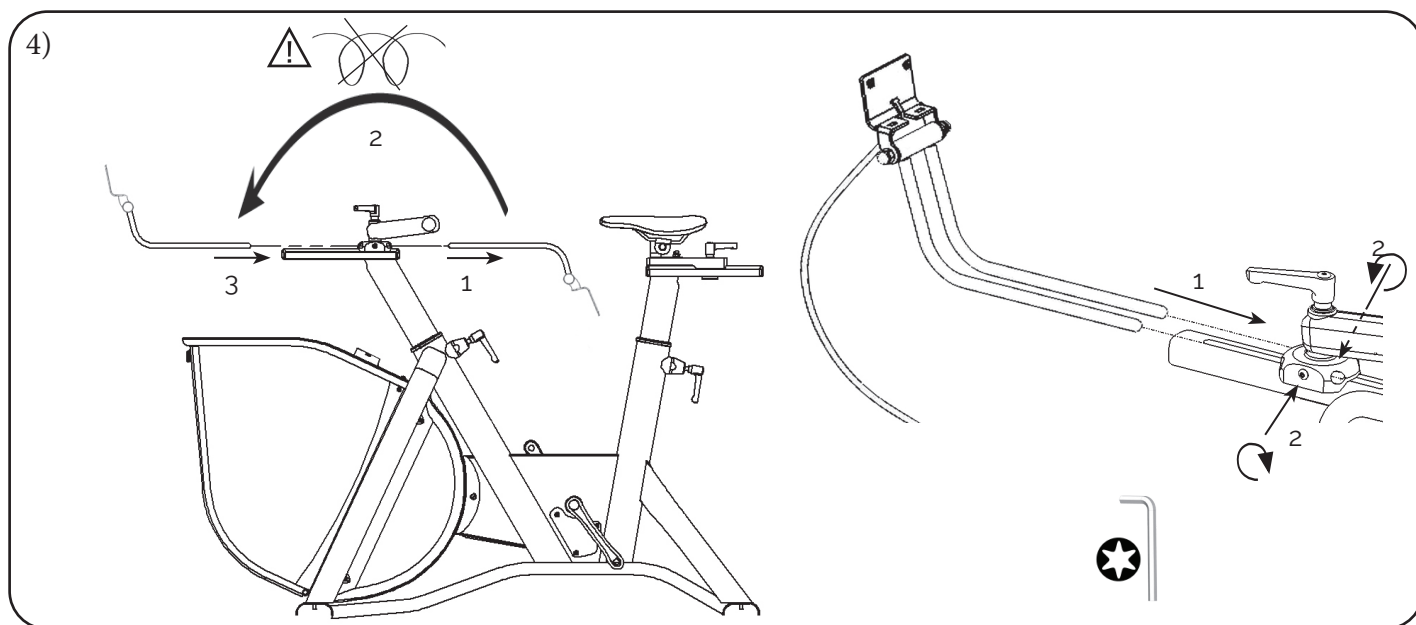3)

MONARK NOVO DISPLAY UNIT

Via the display, you can perform manual training with full precision and performance!

With Monark novo Display Unit, you can control the ergometer via the touch screen. The display shows the current rpm, time, power and pulse (if any chest belt is used).

You can also set up target based training regarding distance, time and calories. If you want to do fitness tests, etc., you can download Monark Test Software from our website, www.monarkexercise.se, and then connect your PC to Monark novo Control Unit on your ergometer.

START TRAINING

The display will start automatically after you have turned on the power to the bike.

Then the display shows the Start view. Now if you want to start, press the "Start" button, the display will show normal mode and the time starts counting.

If autostart is enabled, the time starts count ten seconds after you have started to pedalling, without you have to press "Start". Autostart is set in "Settings general".

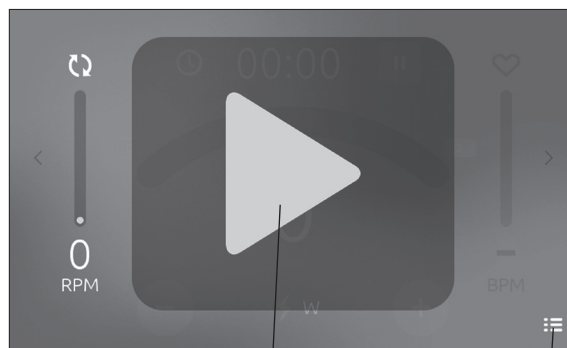
Fig: Start view

Start

Menu

NOTE!

To change workload the rpm has to be at least 30. If rpm is lower, the value is displayed in red. It returns automatically to normal view when rpm is more than 30.

If you want to change the type of workload and the unit, press the icon for the current selection, see section "Workload control".

This is "Normal mode" that can be displayed in two versions, complete and simple. You can switch between them by pressing the arrow keys (change screen) in the middle of the short sides on the display.

NOTE!

Do not expose the electronic display to direct sunlight or extremely high temperatures. Do not use solvents when cleaning, only a dry cloth.

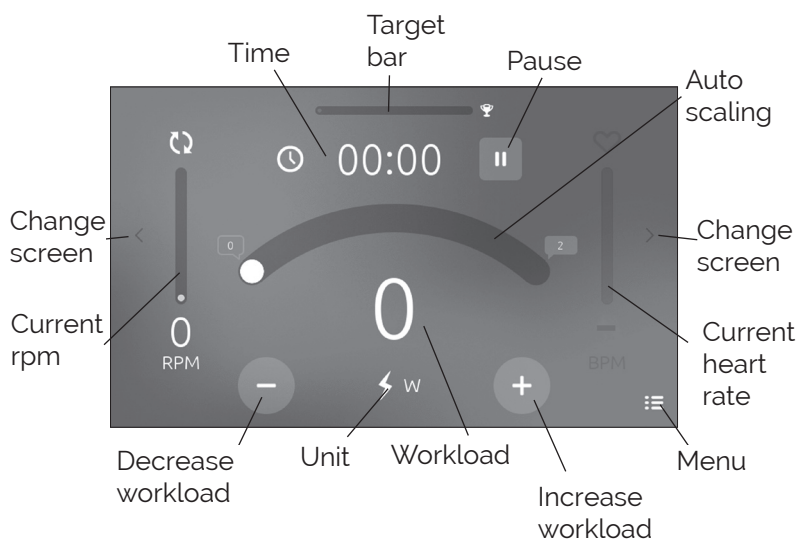
Fig: Normal mode, complete

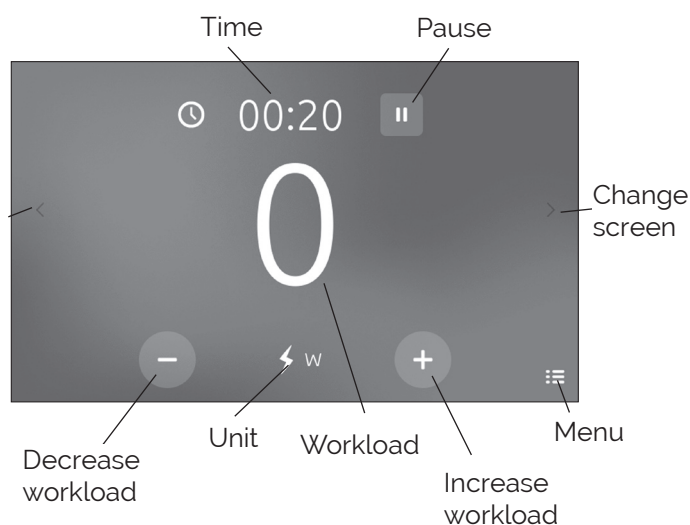
Fig: Normal mode, simple

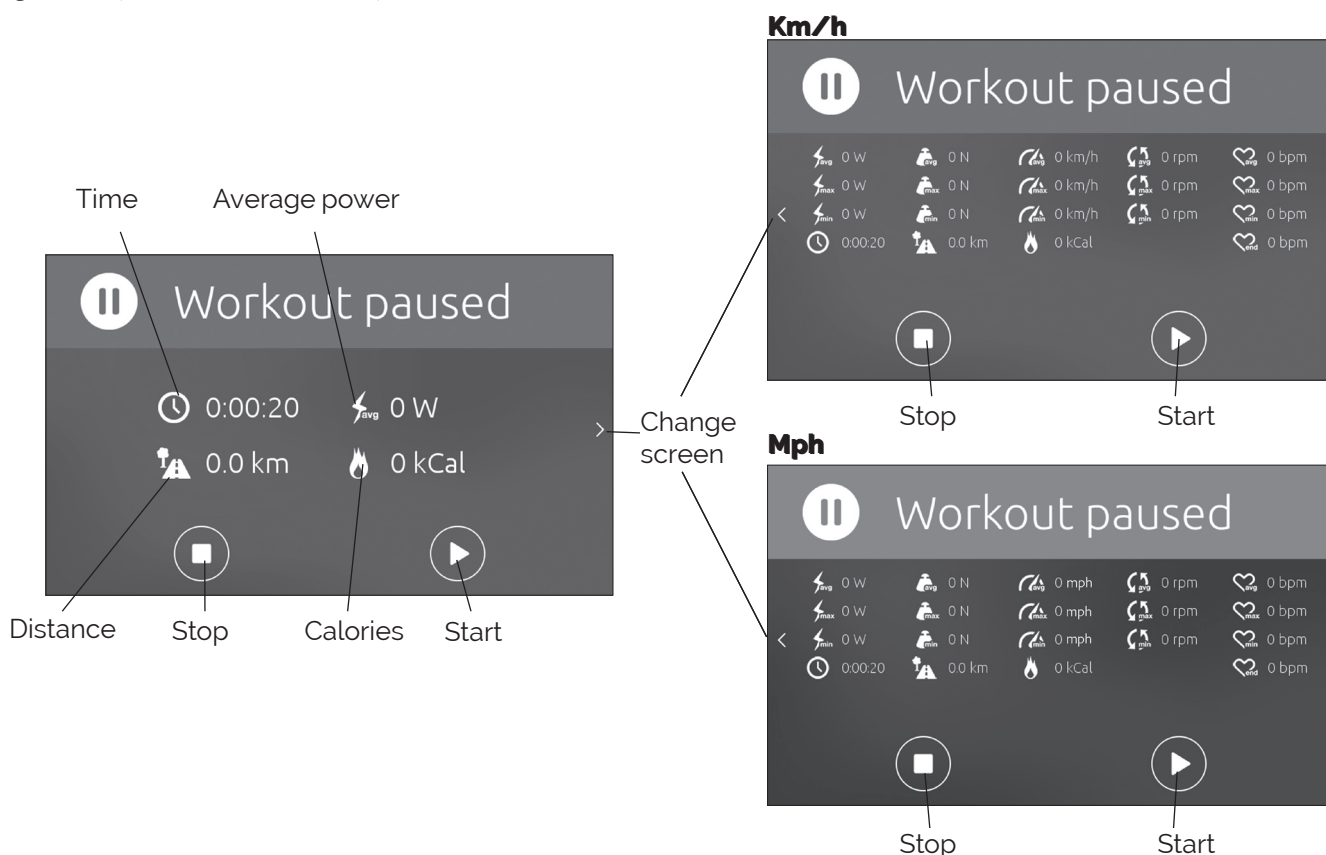
STOP / PAUSE TRAINING

If you want to stop training, press "Pause" or just stop pedalling. After you have pressed "Pause", or about 10 seconds after you have stopped pedalling, the display is changed to "Workout paused", and the workload goes down to "base power".

If you wish to continue the already initiated training, press "Start" and time counting is continuing and the workload returns to the last value. To cancel press "Stop" and the display is reset.

Below you can see how the results is displayed, and you can switch between the two different views by pressing the arrow keys (change screen).

The results can be displayed in both km and km / h or miles and mph, depending on what you selected in the "Settings general" (Metric units: Yes / No).

	Average power		Average workload		Average speed		Average rpm		Average heart rate
	Max. power		Max. workload		Max. speed		Max. rpm		Max. heart rate
	Min. power		Min. workload		Min. speed		Min. rpm		Min. heart rate
	Total time		Total distance		Calories				End heart rate

INTERVAL TRAINING

Use "Start" and "Pause" to do interval training!

Start cycling and select the workload you want, press "Pause" and the bike releasing the workload. Then press "Start" and the bike put on the workload again. The time is not reset, it continues counting.

GEAR LEVER

The workload can be regulated via the gear levers on the handlebars. From the factory, it is set so that on the left side you change in steps of 100 W and on the right side in steps of 10 W. How large the steps should be is possible to set in Monark Test Software (MTS). In MTS you can also change if you prefer to adjust the workload with large steps on the right side or small steps on the left side.

Right lever

On a regular cycle the gear lever on the right side controls the rear derailleur. If you press (1) you switch down and the workload increases. If you press (2) you switch up and the workload decreases.

Left lever

On a regular cycle the gear lever on the left side controls the front derailleur. If you press (1) you switch down and the workload decreases. If you press (2) you switch up and the workload increases.

Fig: Right lever
1) "Down" gear
2) "Up" gear

Fig: Left lever
1) "Down" gear
2) "Up" gear

MENU

In the display is a menu where you can make different settings. You find it by pressing the Menu icon. Then a menu is showed to the right in the display where you can select the following:

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ANT+ settings
(ANT+ Equipment Settings)

General settings
(Settings)

Menu

Target

Heart
rate

ANT+
Equipment
Settings

Settings

You can at any time, even during the exercise, open the menu to e.g. search for a heart rate monitor.

SET A TARGET

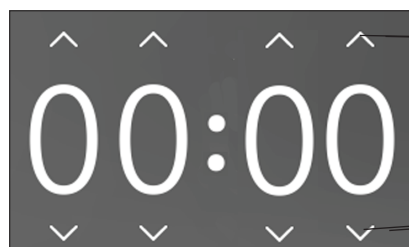
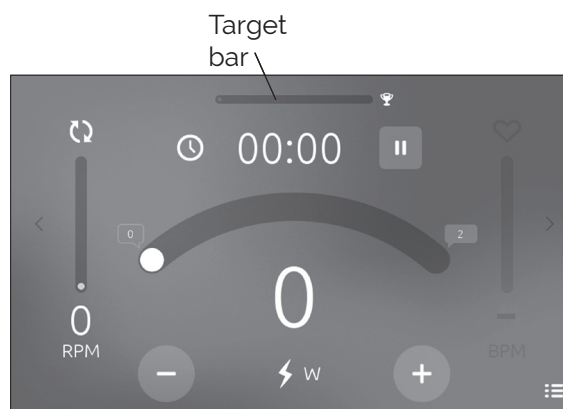
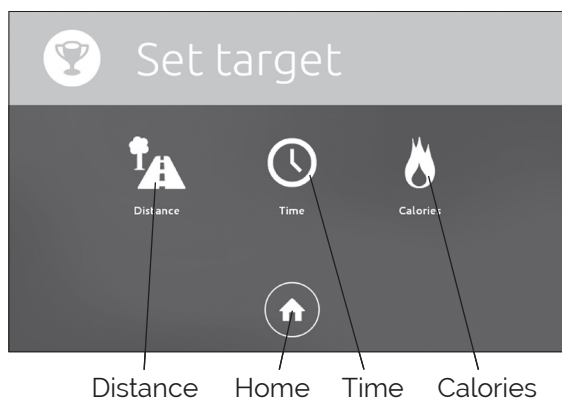
Here you can choose your settings if you want to exercise against a specific target - distance, time or consumed calories.

If you want to train for a specific distance, a specific time or a specific amount of calories consumed, set the value and then the bike counts down / up.

When you train against a specific target, a "target bar" is showed in the display. It shows how far you have left until your set target has been achieved.

The figure below is displayed when you choose "custom" no matter what target you want to set up. When you press the arrow up increases the value from 1, when you press the arrow down decreases the value from 9. At the same time, the value that appears above "home" is changed to show the value you set.

Press, increase value
(1, 2, 3...)

Press, decrease value
(9, 8, 7...)

Set distance target

Here you can choose how long distance you want to train. There are some preset distances, but you can also select "custom" and then you choose a distance. The distance can be displayed in both km and miles, depending on what you selected in the "Settings general" (Metric units: Yes / No).

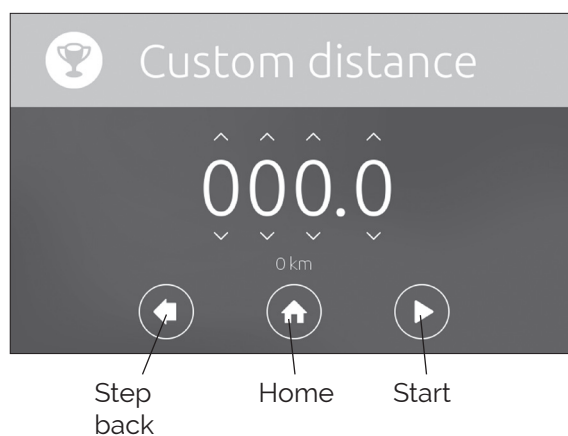
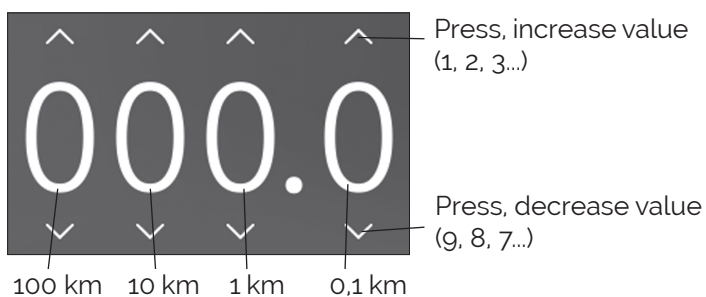
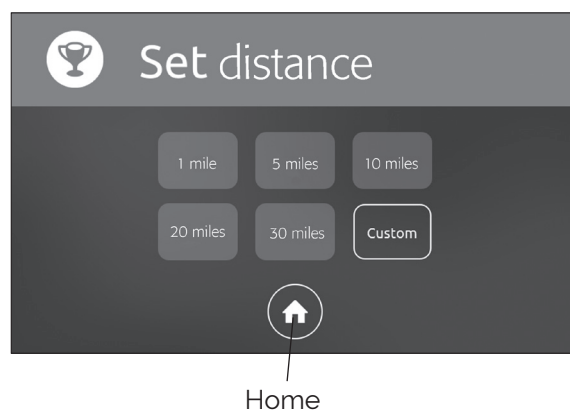
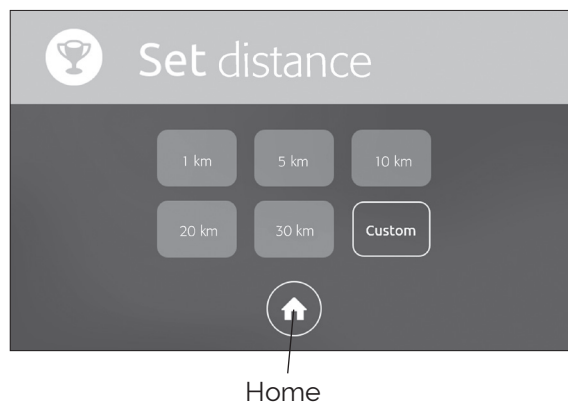
The distance is not exactly, but it is a calculation:

50 rpm = 20 km / h, others rpm linear.

1 rpm $\longleftrightarrow$ 2/5 km / h

1 km / h $\longleftrightarrow$ 5/2 rpm

"Target bar" appears in the display.




Set time target

Here you can choose how long time you want to train. There are some preset times, but you can also select "custom" and then you choose how long time you want to train.

The time in the display will count down to zero.

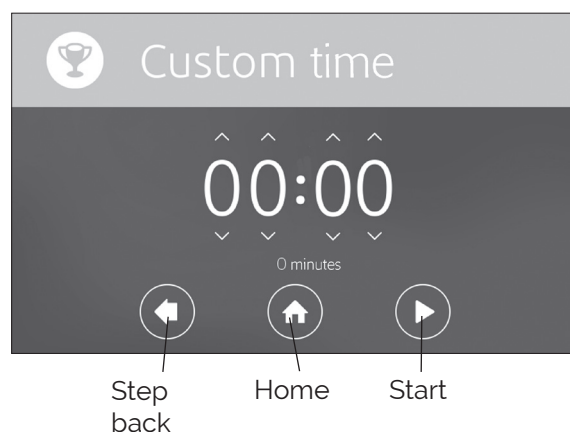
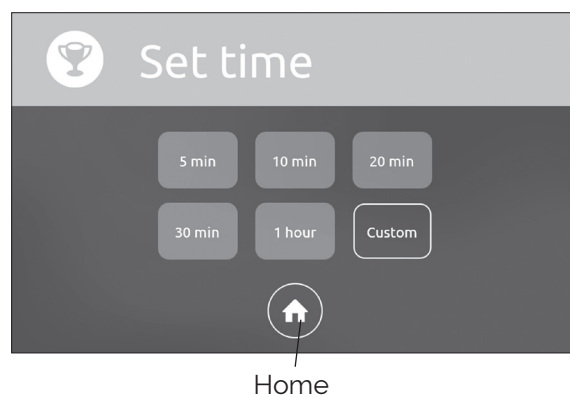
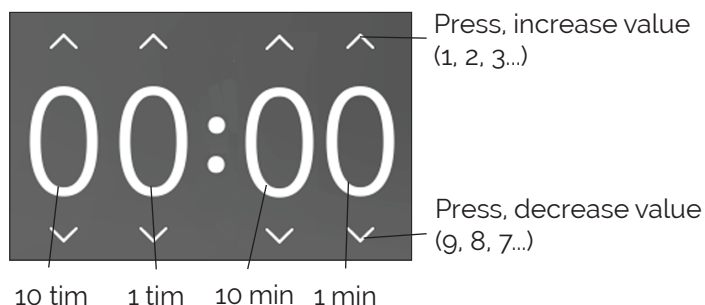
"Target bar" appears in the display.

Set target in kCal

Here you can choose how many calories you want to burn during your training. There are some preset values, but you can also select "custom" and then you choose how many calories you want to burn during your training.

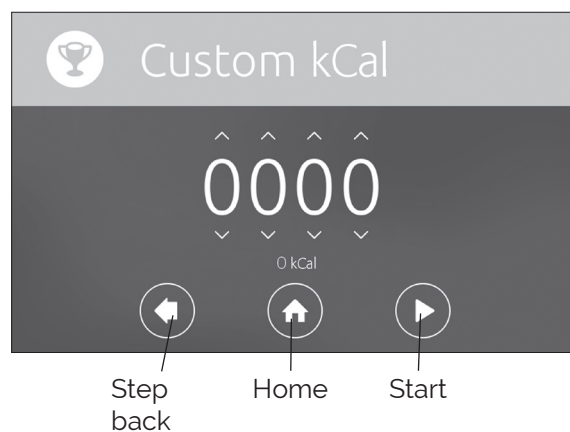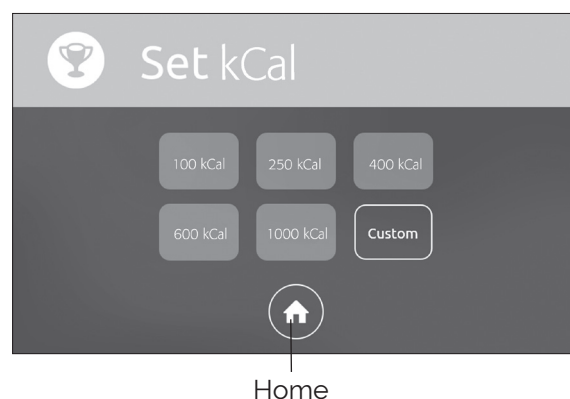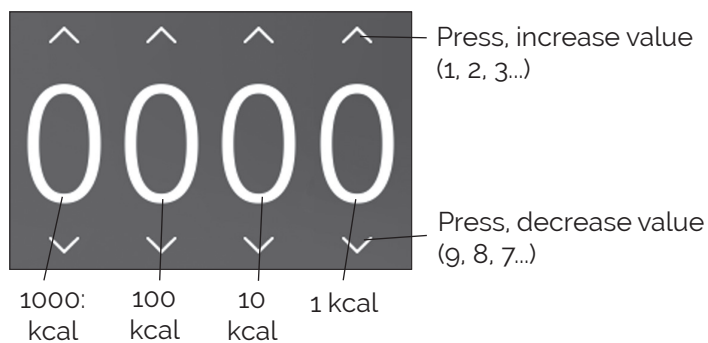
"Target bar" appears in the display.

Exercise target achieved

When you have achieved your target, the display shows "Target achieved". There you can see your results, and you can switch between the two different views by pressing the arrow keys (change screen).

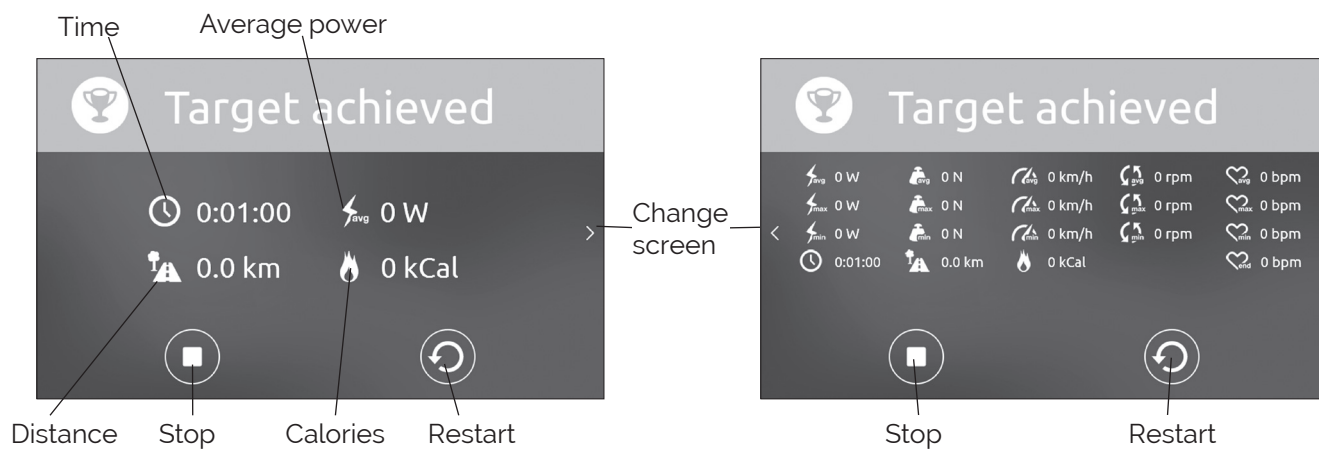The results can be displayed in both km and km / h or miles and mph, depending on what you selected in the "Settings general" (Metric units: Yes / No).

⚡_{avg} Average power
⚡_{max} Max. power
⚡_{min} Min. power
🕒 Total time

👤_{avg} Average workload
👤_{max} Max. workload
👤_{min} Min. workload
👤 Total distance

🏍️_{avg} Average speed
🏍️_{max} Max. speed
🏍️_{min} Min. speed
🔥 Calories

🔄_{avg} Average rpm
🔄_{max} Max. rpm
🔄_{min} Min. rpm

❤️_{avg} Average heart rate
❤️_{max} Max. heart rate
❤️_{min} Min. heart rate
❤️_{end} End heart rate

PULSE SETTINGS

A chest belt is supplied with the ergometer (Bluetooth Smart).

During exercise you can measure your heart rate by using eg. a chest belt. You can use a chest belt that transmits via ANT+ or Bluetooth Smart.

The heart rate can be displayed in the Monark novo Display Unit, in the computer program *Monark Test Software MTS* or in other external device.

Bluetooth SMART (BT-SMART)

When the ergometer and a BT-SMART chest belt are paired together, the chest belt's Bluetooth ID and the current pulse are showed at "Pulse". You can choose to actively lock to that chest belt by pressing "Lock". Then the bike is only searching for that particular heart rate monitor. Press "Reset" if you want to release the lock and lock against a new one or continue to search freely.

You can use multiple BT-SMART heart rate monitors for the ergometer, but you can only lock one at a time.

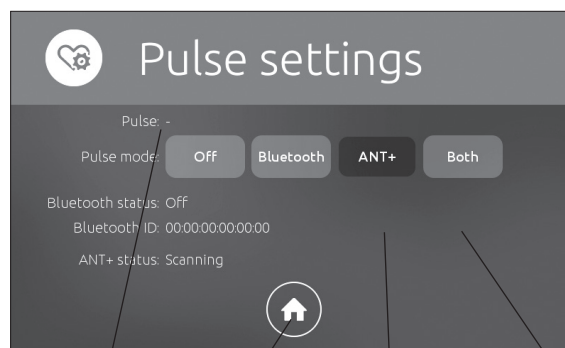
The ergometer locks to the heart rate monitor it finds first (within about 10 m). Therefore, if more devices are used, it is recommended to lock the specific Bluetooth ID to make sure the correct pulse is displayed.

ANT+

(ANT+ is always active)

ANT+ uses "proximity pairing". This means that you should be close to the display at the pairing so that it will start listening to the correct heart rate monitor. When the display and a heart rate monitor are paired, you can be relatively far from without losing the coupling / function.

If you connected to an ANT+ heart rate monitor, it says "Connected" at "ANT+ status:" and the current pulse is displayed at "Pulse".

Current
heart rate

Home

Lock to
BT-SMART/
ANT+

Unlock
BT-SMART/
ANT+

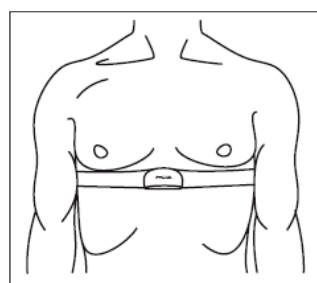
Fig: Placement chest belt

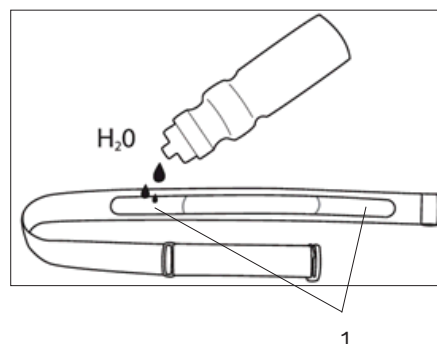
Fig: Moistening the electrodes (1)

NOTE!

You have to make sure that no other heart rate monitors, apart from what is to be locked to the ergometer, are active, that any other device (eg. mobile phone, bicycle computer, etc.), which connects to the heart rate monitor, is active during the process!

ANT+ EQUIPMENT SETTINGS

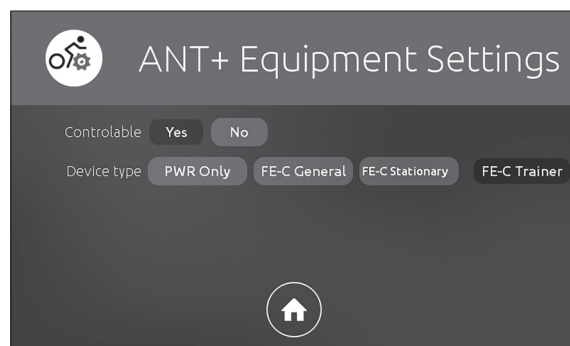
Monark novo supports sending information to certain computer programs, exercise clocks and similar devices via the ANT protocol. Because different programs and devices are programmed differently, there are different settings, where different settings may work depending on which program or device you are trying to connect to. We therefore recommend testing between the various settings until you find someone who works for your particular program or device.


If you try to connect to a computer, you most likely need an ANT dongle. Read about the actual program to find out which products the program's manufacturers recommend.

When testing between the different buttons, we recommend starting from the right. The buttons are in increasing order of how much functionality is supported. The *FE-C Trainer* thus supports most functions and *PWR Only* has the least functions. However, *PWR Only* works with more units.

All variants of *FE-C* support to control the resistance of the bike from a program or device. *PWR Only* does not support steering. Controllable tells the program or device whether or not the bike should be considered controllable. In most cases, you can leave it in the "Yes" position.




GENERAL SETTINGS

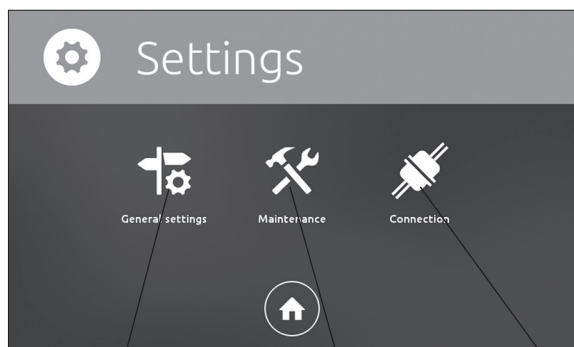General settings

Maintenance

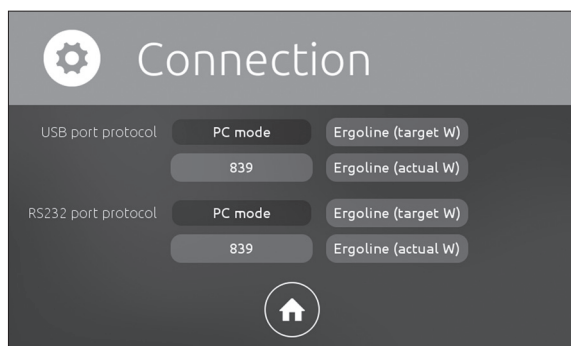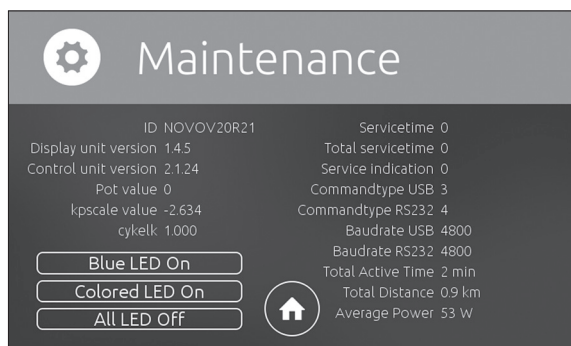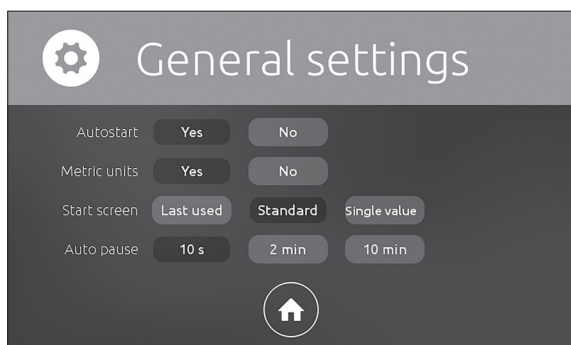Connection

General settings

Auto Start	Sets whether to start timing when the user starts pedalling or if the user is required to press “Start” on the screen.
Metric units	Specifies whether it is km or miles.
Start screen	Specifies the type of screen to be displayed the first time an training is started after restarting the cycle.
Auto pause	How long it takes before training is paused if the user stops pedalling.

Maintenance

Commandtype USB	Here you can see which protocol the USB port is configured for
Commandtype RS232	Here you can see which protocol the RS232 port is configured for

0 = Standard, PC mode

2 = Ergoline compatible (target W, recommended for “er900”)

3 = Ergoline compatible (actual W, alternative to above)

4 = 839 compatible (for older devices)

Total Active Time	Total time in use
Total Distance	Total distance
Average Power	Average load

Connection

Set command type in the display

The bike’s two free ports, USB and RS232, can be set in different command types to fit different standards and external equipment. You can now do this directly in the novo display.

- Go to “Menu”
- Select “Settings”
- Select “Connection”

In the Connection window you can choose which protocol you want the different ports to use, the default for both is PC mode.

To see which port setting is current, go into the Menu, select Settings and then Maintenance.

- Go to “Menu”
- Select “Settings”
- Select “Maintenance”

CHANGE WORKLOAD UNITS

If you press the symbol for units a menu appears, "Set unit", where you can change which workload unit you want to use.

The workload unit you can choose is: l / min, kpm / min, Watt, Newton and kp.

Rpm-independent - l / min (calculated)

- kpm/min

- Watt

Rpm-dependent - Newton

- kp

When you have chosen which workload unit you want to use, the display returns to "Normal mode". If you press "Home" the display returns to normal mode without any changes.

If you press "Locked mode" in the view "Set unit" you lock the display to show the current rpm instead of the current workload, see *Fig: Locked mode*. In this mode, the target bar is also showed. You can also choose to display only the current rpm with only a digit, see *Fig: Locked mode, simple*. To cancel the "Locked mode", press the "Locked mode" and then you will return to "Set unit". There you can choose optional workload unit. When you have chosen unit the display returns to normal mode.

If you are in the "Locked mode", press "Menu", you can choose "Settings pulse" or "Settings general", see further details under each title. .

You can always, even during exercise, change control mode.


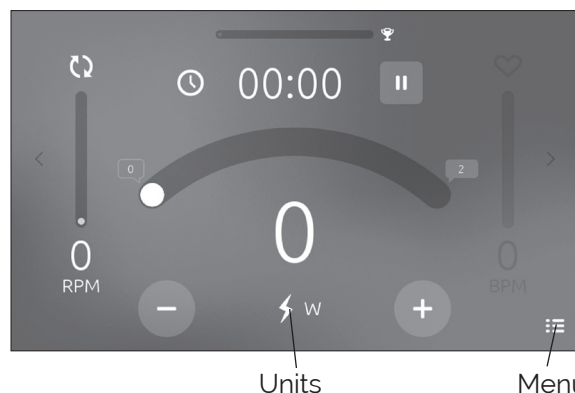
Fig: Normal mode

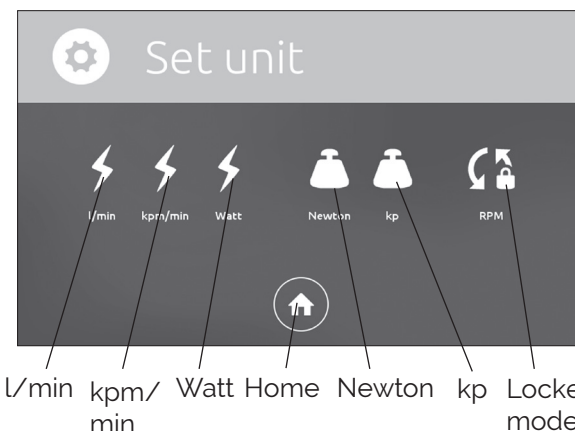l/min kpm/min Watt Home Newton kp Locked mode

NOTE!

The text turns red when the set power not can be reached with the current rpm with maximum workload. Please reduce the workload level or pedalling faster (increase rpm) until the text turns white again.

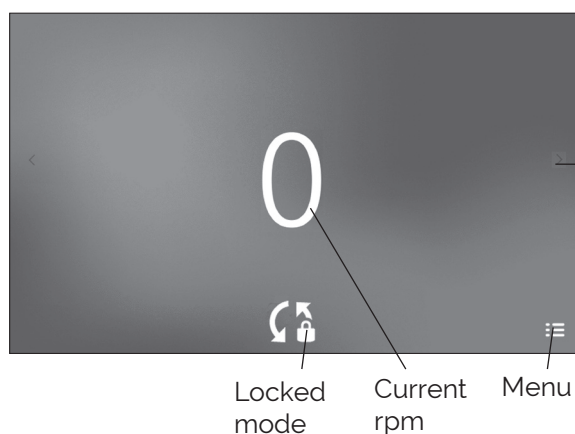
Fig: Locked mode, simple

Change screen

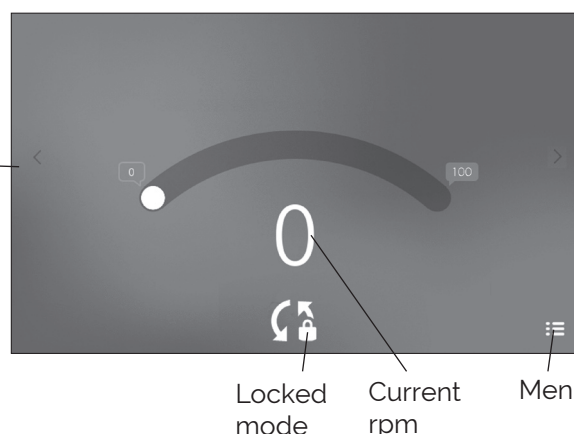
Fig: Locked mode

MONARK NOVO CONTOL UNIT

Connect to external Control Unit

No tests can be performed via Monark novo Display Unit. If you want to perform tests you have to connect your bike to a PC, via the Monark novo Control Unit, and use a special test software. A suitable one is available to download from our website: www.monarkexercise.se.

The bike do not need to be switched off when connecting external components, but it is recommended, to prevent transmission of incorrect data.

Be careful when connecting different types of external devices to prevent flash-over and subsequent injury. The user must ensure that the correct cables are used, otherwise you risk serious injury or damage on the device.

Connect external device via RS232 and USB

Make sure the power adaptor is connected to the bike
Turn off the power switch on the bike.

Both RS232 port and USB-B port are located on Monark novo Control Unit behind the left side front cover. Open the front cover to access Monark novo Control Unit. You can pass the cable you want to connect through the cable inlet in the left side cover. Remember to lock the cable in the strain relief.

Connect the RS232 / USB-B first to Monark novo Control Unit in the bike and then to the external device. Set the bike power switch to "on" and then start the external device.


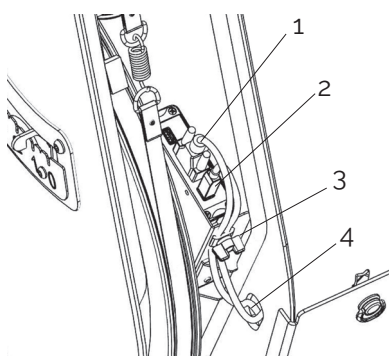
Fig: Connections

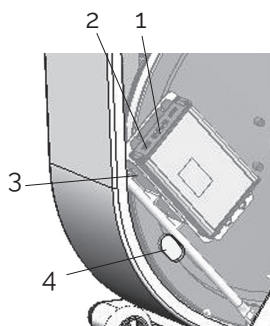- 1) RS232 port
- 2) USB-B port
- 3) Strain relief
- 4) Cable inlet

The side cover is transparent in the figure

Communication protocol (command type)

The bike's two free ports can be set in various command types to suit different standards and external equipments. The four available command types are:

0: Standard command set or "PC-mode"

Command type "0" is default for both of the free ports. As default the baud is 4800, but can be changed by external control. The protocol is used by Monark Exercises softwares and some external controllers that are prepared to fully benefit from the advanced capabilities of the bike.

2: Ergoline compatible (recommended for "er900")

The command type is used when the bike is controlled by an external device that uses Ergoline's "er900" protocol, baud rate is locked to the 4800. If the external device "asks" the bike about current power, the bike responds with the set power. This setting is recommended before the command type 3 which removes the risk of the device trying to "chase" a value.

3: Ergoline compatible (alternative)

The command type is used when the bike is controlled by an external device that uses Ergoline's "er900" protocol, baud rate is locked to the 4800. If the external device "asks" the bike about current power, the bike responds with the set power.

4: 839 compatible

This command type is very similar to command type "0", the only difference is that if an external device requests the ID, the bike responds as if it was an 839-bike of newer model. The setting is used for older external control devices that are not yet adapted to NOVO series (backwards compatible).

All commands for external devices are active regardless of the command type that is selected in addition to the small differences presented above. The parts in Ergoline protocol that handle blood pressure and its parameters are not implemented.

The two free ports can have different settings.

When connecting to multiple devices, it is important to ensure that one device does not sets a new target without the other devices note it (master - slave).

SCALE ADJUSTMENT

First check that the brake belt is loose, see *Fig: Control loose brake belt*. If not, move the pendulum to 4 kp and hold it there a while to loosen it. Then check that the pointer on the pendulum is aligned with the 0-index on the scale. If not, you need to adjust the scale. Loosen the pole screw (do not unscrew it completely), and adjust the scale so that the 0-indices and indicators are consistent. Tighten the pole screw again as the scale is locked into position. See *Fig: Scale adjustment*.

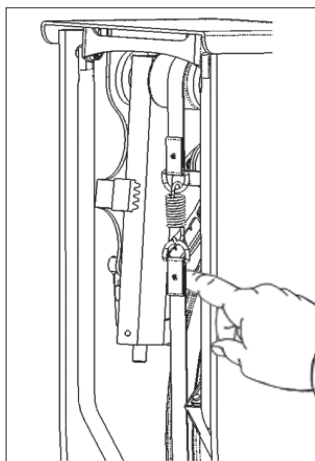
Fig: Control loose brake belt

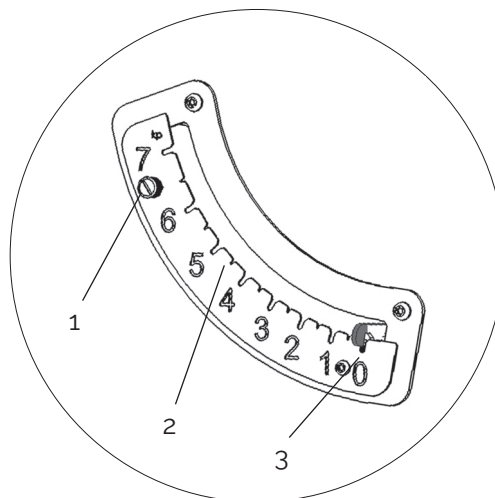
Fig: Scale adjustment

- 1) Screw
- 2) Kp-scale
- 3) 0-index

CALIBRATION PENDULUM

All test- and exercise cycles are calibrated in the factory, but a calibration of the pendulum can still be done to verify this.

Check the calibration of the pendulum once a year or when needed.

Hang 4 kg calibration weight in the spring. Then the pendulum pointer should stop at 4 kp on the scale. If is not, the adjustment weight in the pendulum must be adjusted, see *Fig: Pendulum*.

If the pendulum stops too high, the adjustment weight must be moved outwards / downwards in the pendulum, and if the pendulum stops too low, the weight must be moved inwards / upwards in the pendulum. Repeat until the pendulum stops at 4 kp.

When finished, remove the weight and replace the brake belt in the spring. Then close the front cover.

After calibrating the pendulum, you must do an electronic calibration, see section.

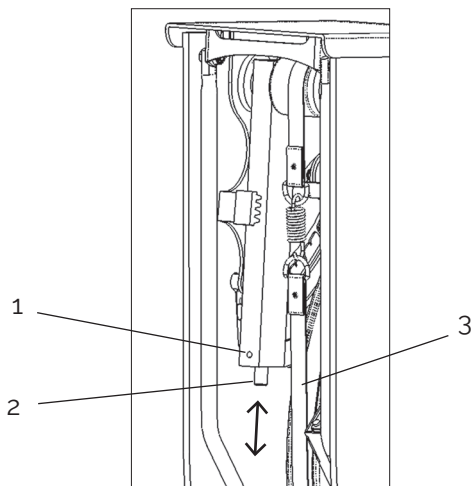
Fig: Pendulum

- 1) Adjustment screw
- 2) Adjustment weight
- 3) Brake belt

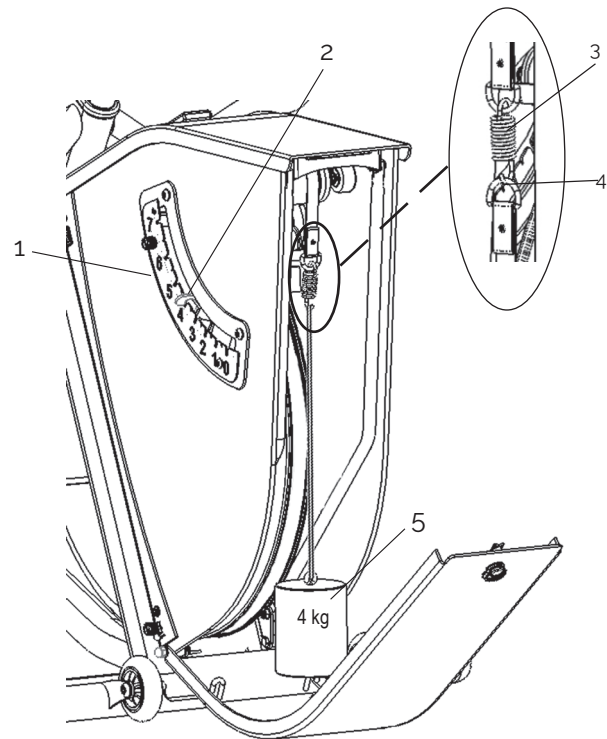
Fig: Calibration

- 1) Scale
- 2) Pendulum pointer
- 3) Spring
- 4) Hook to hang the weight in
- 5) Calibration weight 4 kg

NOTE!

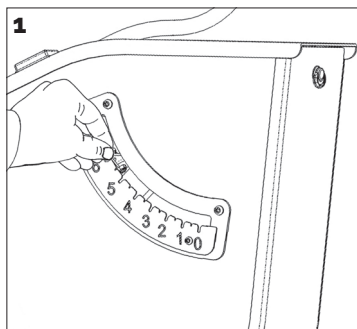
The flywheel must be completely stopped before the weight is hung on!

ELECTRONIC CALIBRATION - WITH THE PENDULUM

Before calibrating the ergometer, make sure that the brake belt runs freely on the freewheel. If not, lift the pendulum to 4 kp and hold it for a few seconds to release the load (the power must be on the ergometer).


NOTE!

The pendulum must be kept still at the different positions. This is done by pressing down the pointer into the groove on the scale at each kp value.


1 Start the calibration

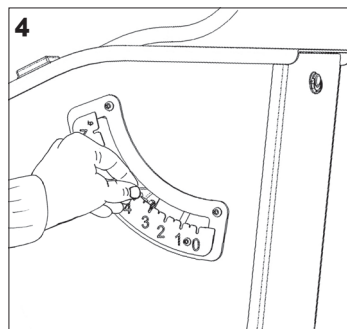Turn off the power to the ergometer. Lift the pendulum to 6 kp and hold it there. Connect power to the bike. After about five seconds, 10 quick beeps are heard (= calibration mode). Move down the pendulum to 0 kp and then follow the pictures.

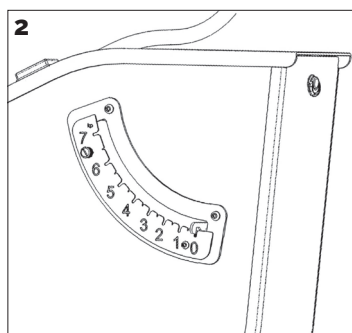
Hold the pendulum at 4 kp and wait for a beep. Wait for the beep tone to stop before you move the pendulum to next calibration point.


2 Zero position

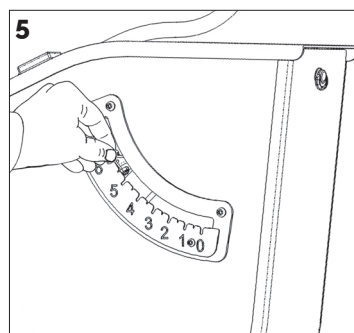
Hold the pendulum at 0 kp and wait for a beep. Wait for the beep tone to stop before you move the pendulum to next calibration point.

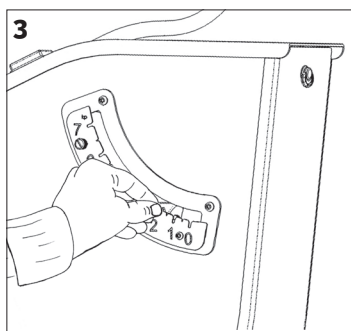
Hold the pendulum at 6 kp and wait for a beep. Wait for the beep tone to stop before you move the pendulum back to 0 kp. The calibration is complete.

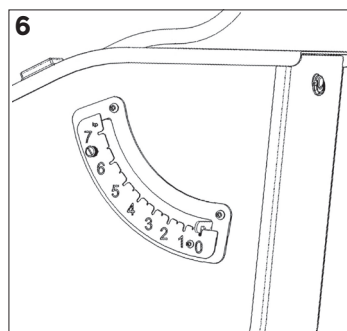Hold the pendulum at 2 kp and wait for a beep. Wait for the beep tone to stop before you move the pendulum to next calibration point.

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Hold the pendulum when you move it down to 0 kp again.

MONARK EXERCISE

FOR LIFE AND PERFORMANCE

Version 2112

Art. Nr: 7950-410SE